# EASY RIDE

**1P650**

## PATIENT

<b>FR</b>	Notice d'Utilisation	<b>FI</b>	Käyttöohjeet
<b>EN</b>	Instruction for use	<b>PL</b>	Instrukcja użytkowania
<b>DE</b>	Gebrauchsanweisung	<b>CS</b>	Návod k použití
<b>IT</b>	Istruzioni per l'uso	<b>SK</b>	Návod na používanie
<b>ES</b>	Instrucciones de uso	<b>HR</b>	Upute za uporabu
<b>PT</b>	Instruções de utilização	<b>RU</b>	Инструкция по использованию
<b>NL</b>	Gebruikershandleiding	<b>UK</b>	Інструкція з використання
<b>DA</b>	Brugervejledning	<b>JA</b>	取扱説明書
<b>NO</b>	Bruerveiledning	<b>ZH</b>	使用说明
<b>SV</b>	Bruksanvisning	<b>AR</b>	تعليمات الاستخدام

# EASY RIDE – 1P650

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1P65098  
2021-07

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Patient information leaflet

1P65098  
2021-07

Please read before use in the presence of your Prosthetist/Orthotist

## 1. COMPONENTS INCLUDED

Designation	Ref.	Included/Sold separately
Knee	1P650	Included
Air pump	1P65012	Included
M6x14 screws (x2)	WMBT6014Z	Included
Torx T30 key (x2)	WKT030	Included
Pin removing tool	1P65019	Included
Female pyramids	1K185 and 1K66	Sold separately
Cover kit	1P653	Sold separately

## 2. DESCRIPTION, PROPERTIES AND MECHANISM OF ACTION

### A. Description

A single-axis, multi-sport knee that can withstand high impacts, fitted with a state-of-the-art air/oil shock absorber, a distal connector and a proximal connector that can rotate.

The wearer can easily modify the knee position and settings to adapt it to the type of activity to be performed: SLIDING or BIKING  
Male pyramid adapters provide its proximal and distal connection. These pyramids are designed to connect to a female pyramid. The male pyramids provided can be replaced by female pyramids, which can be ordered separately.

### B. Properties


Ref.	1P650	
Weight	1200 g	
Position	SLIDING	BIKING
Pre-flexion	12°	0°
Maximum flexion	50°	130°
Maximum patient weight (including carried load)	125 kg	

This device has undergone a one-million-cycle test, in accordance with the conditions described in standard NF EN ISO 10328 at a P6 load level (i.e. 125 kg). The service life depends on the activity performed and on how often and how intensively it is used by the patient.

## 3. INTENDED USERS/INDICATIONS

This medical device is supplied to healthcare professionals (prosthetists/orthotists) who then instruct the patient in how to use it. The prescription is drawn up by a doctor. The doctor works with a prosthetist/orthotist to assess whether the patient is suited to using the device.

⚠ This device should be used on ONE PATIENT ONLY. It must not be reused on other patients.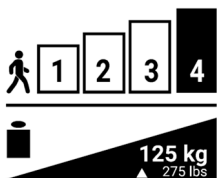

This device is exclusively intended to be used as prosthetic equipment by amputees wishing to engage in sports activities. It is specifically recommended for very active patients (L4) and allows them to practice sports such as surfing, snowboarding, cycling, mountain biking, roller skating, ice skating and paddleboarding.

⚠ **This device is not intended to be used for walking.**

Maximum weight (including carried load): 125 kg

⚠ **Not suitable for children.**

## 4. CLINICAL BENEFITS

The device allows:


- The patient to practice many summer and winter sports (road cycling, skateboarding, surfing, snowboarding, etc.)
- Adjustment of the degree of shock-absorption both in flexion and in extension
- Adjustment of the resistance to flexion and of the extension assist force (by inflating the pneumatic system)
- Customisation to different sports by changing the position of the pin which secures one end of the shock-absorber.

## 5. OPERATION/INSTRUCTIONS FOR USE

### A. Change of position setting

To switch between the SLIDING (for sports performed on a board, skis, skates, etc.) and BIKING positions, move the pin that secures the upper end of the shock absorber from position 1 to position 2, as follows:

- Unscrew the two screws at the two ends of the pin using the two Torx T30 keys supplied.
- Remove the pin using the pin removing tool supplied.
- Insert the pin in the desired position.
- Refit the two screws with their washers, apply a drop of XC048 low-strength thread-locking compound, and tighten the screws to a torque of 6 Nm.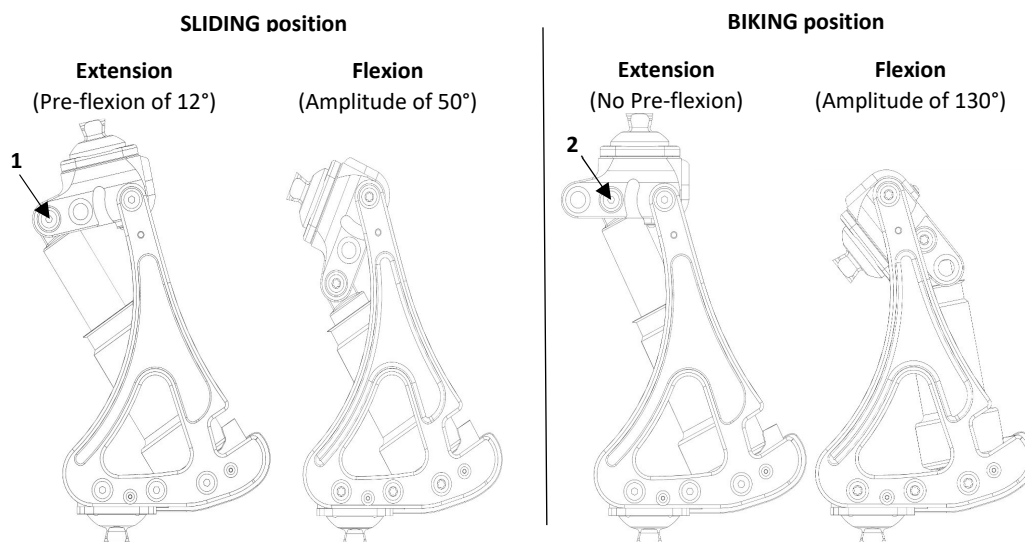

- ⚠ The screws must be tightened and untightened with a torque wrench fitted with a Torx T30 bit.
- ⚠ If the screw head becomes deformed, replace the screw with one of the two spares supplied.
- ⚠ The maximum flexion stops in the BIKING position are only present to protect the shock-absorber; they must not under any circumstances be subjected to stresses regularly and repeatedly.

### B. Selecting the position and settings for different activities

The shock-absorber fitted offers three setting controls:

- **BLUE control** (FIRM / MEDIUM / OPEN) sets the compression speed to slow, medium and fast respectively.
- **BLACK control** (ADJUSTING TRAIL MODE) used to fine tune the compression speed, in OPEN mode only, with 3 positions available: 1, 2 and 3 from soft to firm.
- **RED control** (REBOUND) sets the extension speed.

### C. Adjusting shock-absorber pressure

Using the pump provided, pressurise the shock-absorber to the required pressure.

If the pressure is too high, reduce it using the air release button on the pump.

Only used the pump supplied or a pump fitted with a 20-bar gauge suitable for FOX shocks.

The combinations of positions and settings indicated below are provided as a guide only and may vary as a function of the activities performed and the patient's weight:

#### Sporting activities:

- Pin position:**
  - BIKING: road cycling, cross country MTB, downhill MTB and motocross
  - SLIDING: all other activities (on a board, skis, skates, etc.)
- Flexion setting:**
  - OPEN (blue control)
- Extension settings:**
  - REBOUND generally set to maximum and adjusted as required (red control)
- Pressure:**
  - 0 bar: Road cycling, cross country MTB
  - 1 to 3 bars: Surfing, paddleboarding, waterskiing, wakeboarding
  - 4 to 6 bars: Downhill MTB, snowboarding, piste skiing

- 5 to 10 bars: Fitness training, racket sports, combat sports, motocross
- ⚠ The air pressure in the shock-absorber must never exceed **20.6 bars**.
- ⚠ It is essential to **release all the air** from the shock-absorber when travelling at high altitude (e.g. in an aeroplane).
- ⚠ **Refit the cap** to the valve after adjusting the air pressure.

**Moving between activities/getting past obstacles/need for safety:**

1. Flexion setting:
  - FIRM (blue control)

## 6. DETECTING ANY MALFUNCTIONING

- ⚠ If you notice that the device is behaving abnormally or if its characteristics feel different, or if it has been badly knocked, consult your prosthetist/orthotist.

## 7. WARNINGS, CONTRAINDICATIONS, SIDE EFFECTS

### A. Warnings

- ⚠ When using the knee again after a period of inactivity, traces of oil may appear. Wipe them away with a soft cloth.
- ⚠ The shock-absorber has been prepared specifically for use as part of the knee and must not under any circumstances be disassembled or used for any other purpose.
- ⚠ Leave the shock-absorber to cool down after use; be aware that it may be hot enough to burn skin.
- ⚠ If the temperature of the shock-absorber changes significantly its behaviour may vary.
- ⚠ The shock-absorber is unpressurised when delivered to eliminate any problems while in transit.
- ⚠ It is possible to trap a finger or pinch clothing in the knee joint. To avoid the any injury caused by the movement of the joint, make sure that no-one's hand or fingers is near the mechanism.

### B. Contraindications

- ⚠ Never lubricate the knee pins; this might cause them to degrade rapidly.
- ⚠ If a load is being carried, the operation of the knee may be disrupted.
- ⚠ The warranty does not cover damage or degradation caused by misuse, unsuitable alignment, use in an unsuitable environment or any other inappropriate use.
- ⚠ If a malfunction occurs or if premature wear or traces of rust are detected, consult your prosthetist/orthotist as soon as possible
- ⚠ Never leave this device near a source of heat since it might burn or release toxic fumes.
- ⚠ The use of solvents is prohibited.

### C. Side effects


There are no side effects directly associated with the device.

Any serious incident that occurs which relates to the device must be reported to the manufacturer and to the competent authority of the member state in which the incident occurred.

## 8. MAINTENANCE, STORAGE, DISPOSAL AND SERVICE LIFE

### A. Maintenance/cleaning

- ⚠ Check regularly the condition of the maximum flexion stop screws
- ⚠ Check regularly that the M6 screws are tight (tightening torque of 6 Nm).
- ⚠ If a malfunction occurs or if premature wear or traces of rust are detected, consult your prosthetist/orthotist as soon as possible
- ⚠ You can clean the knee using a damp sponge
- ⚠ Clean the knee after exposing it to potential contamination (water, mud, etc.) and if it is immersed.
- ⚠ After immersion in sea water, rinse carefully in fresh water and then dry completely.
- ⚠ Do not use a solvent or other aggressive product when cleaning
- ⚠ To clean the shock-absorber, refer to the manufacturer's instructions provided with the knee.

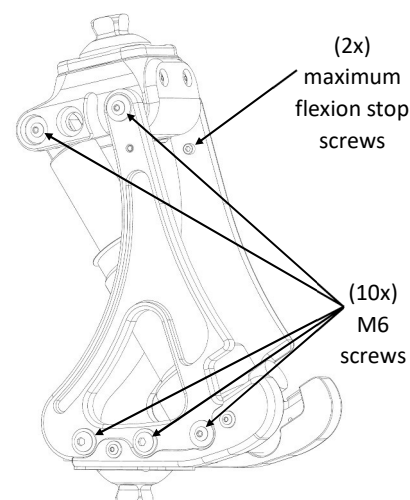### B. Storage

- ⚠ Service and storage temperature: -25°C to +40°C
- Relative humidity of the air: no restrictions

### C. Disposal

The various components of this device are considered as special waste: aluminium, stainless steel pins and screws, titanium connectors, plastic bushes and oil. They must be treated in accordance with current, applicable legislation.

### D. Service life

The knee must be serviced after every 100 hours of use by your prosthetist/orthotist (i.e. every year if used at moderate activity levels or every 6 months at a higher level of activity).

## 9. DESCRIPTION OF THE SYMBOLS

	Manufacturer		Identified risk		CE marking and year of first declaration of conformity
--	--------------	--	-----------------	--	--

## 10. REGULATORY INFORMATION

This product is a CE-marked medical device that is certified as conforming with regulation (EU) 2017/745

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