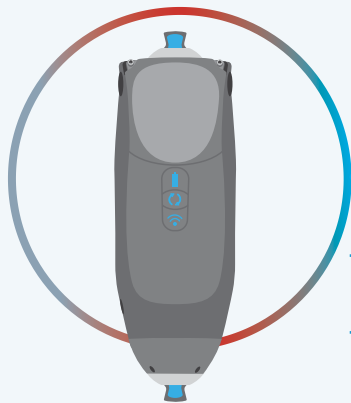
PROTEOR
QUATTRO[®]
MICROPROCESSOR KNEE

► **Patient Quick Start Guide**

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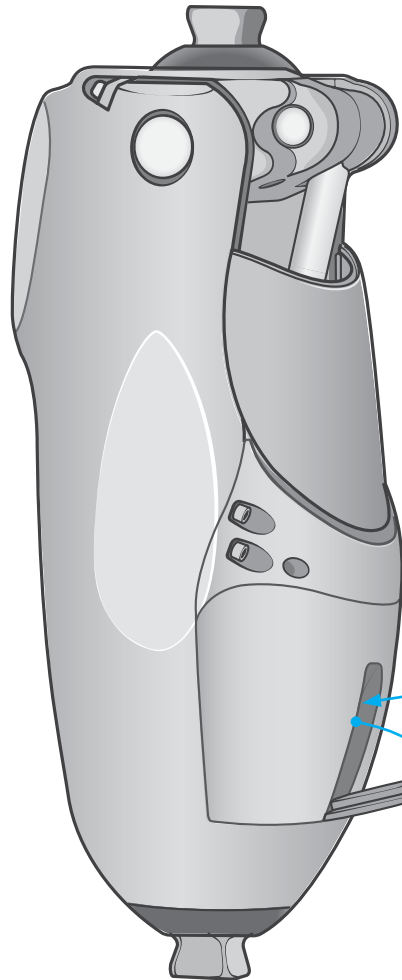
MICROPROCESSOR KNEE


1. Open the charge port.

2. Insert the charger or battery pack into the charger port.

3. Connect to a proper power source.

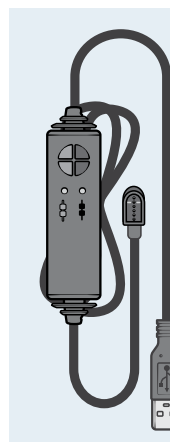
4. Charge time:
50%: 1–1.5 hours
100%: 3 hours

! WARNING!

Do not submerge the portable charger in water while charging.

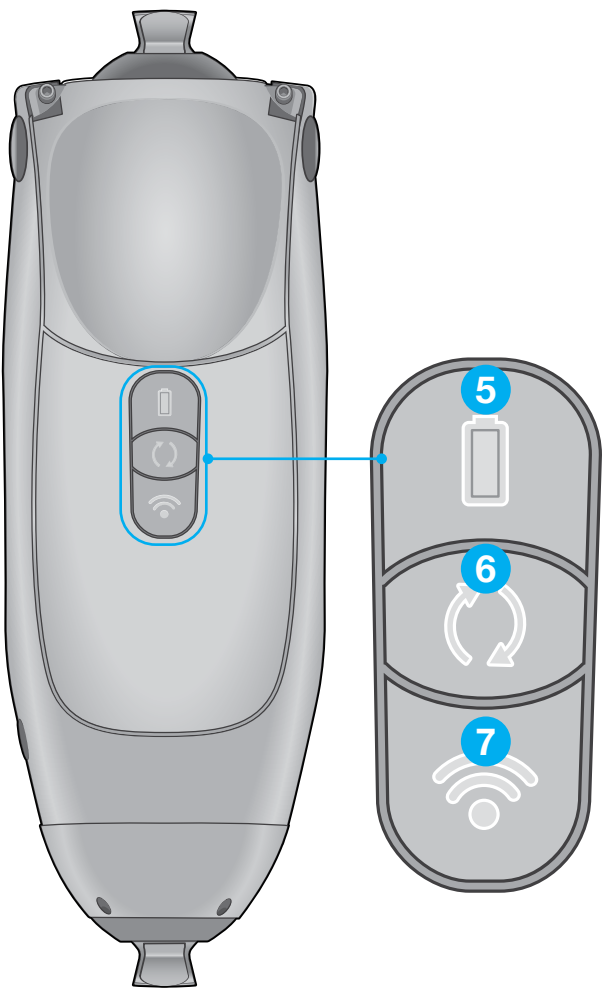KEYPAD

Button	Status	Charge level	LED description
5 Battery Status	Fully charged	51–100%	
	Charged	21%–50%	
	Low	11–20%	
	Very low	1–10%	
	Knee reboot	Hold 6 seconds	
Button	Function	Status	LED description
6 Change Mode	Check activity Press and release	Unlocked	Walking: Alternate:
		Locked	
	Enable/disable lock Press twice	Lock engaged	
		Locked engaged, with very low battery	
		Locked disengaged	Previous LED activity resumes
	Toggle between Walking and Alternate Long press	Current activity: Walking	
		Current activity: Alternate	
		Current activity: Walking Alternate activity not enabled in patient app or unsafe to switch activities	
Button	Function	Description	LED description
7 Wireless	Pair with knee	Pairs with devices	None

LED Key	
Solid green	
Blinking green	
Solid red	
Blinking red	
Blinking blue	
Solid white	
Slow blinking white	
Fast blinking white	

CONNECT

TOOLBAR

Activities

Adjust

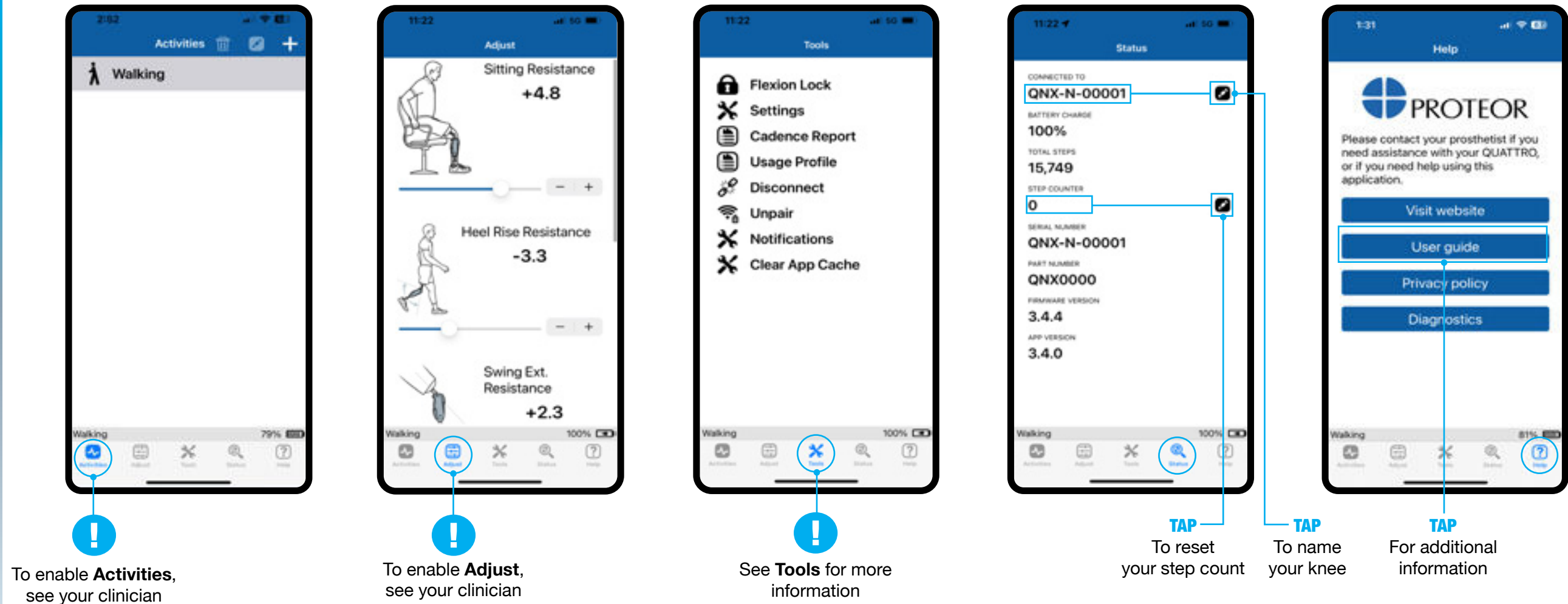
Tools


Status

Help


TOOLS

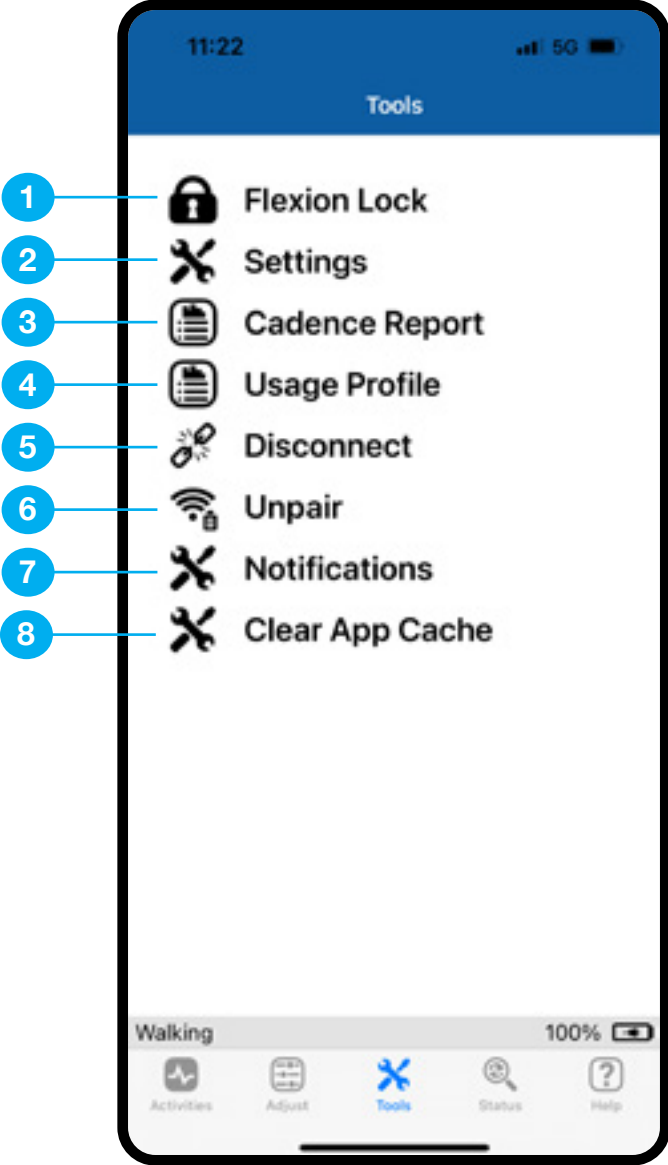ICON	TAP TO	TO USE
1 Flexion Lock	Allows the QUATTRO to block any further bending but allows for extending when turned on.	Press the Mode button twice to turn on , or use the slider in the App.
2 Settings	Disables Sitting Mode .	Turn Enable/Disable via the App slider.
3 Cadence Report	Run a Cadence Report and email to a recipient of your choice.	Complete the required fields. Send email to recipient.
4 Usage Profile	Allows you to run reports for; total steps taken, total number of steps taken on flat ground, up stairs, down stairs, up ramps, down ramps and backwards.	Complete the required fields. Send email to recipient.
5 Disconnect	Disconnect the Freedom Innovations App from the QUATTRO knee.	Confirm popup.
6 Unpair	Unpair your device from the QUATTRO knee.	Confirm popup
7 Notifications	Choose the types of notifications that are received.	Turn Enable/Disable via the App slider.
8 Clear App Cache	Clear App cache memory to improve function.	When the function of the App becomes unstable.


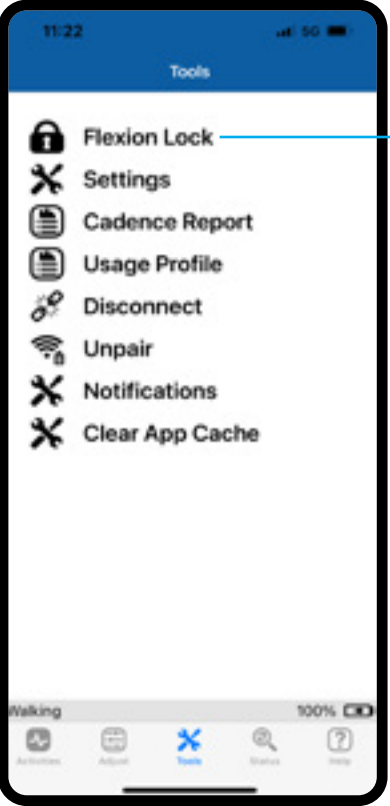
MODES AND LOCKS

Feature: Flexion Lock
Function: Knee will lock at the current angle when Lock is engaged via the Keypad or the App.
Knee can still straighten, but once straightened it will relock at that new angle until Lock is disengaged via the Keypad or the App.
Use:
Engage: Double click the **Mode** button or turn the slider green in the App. The solid white light remains on as long as the Flexion Lock is engaged.
Disengage: Double click the **Mode** button or turn the slider grey in the App and the solid white light turns off.

TIP!

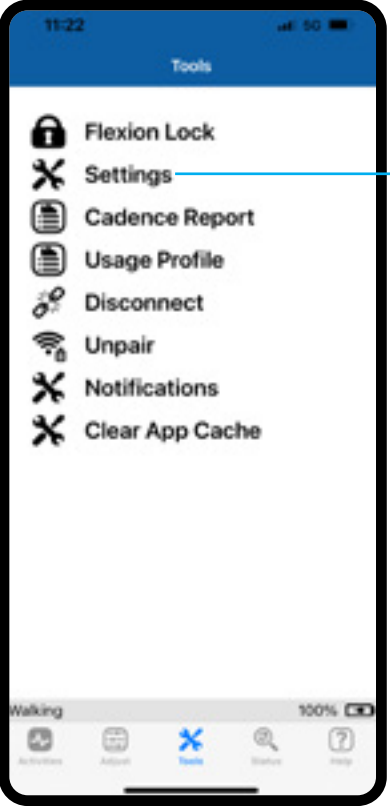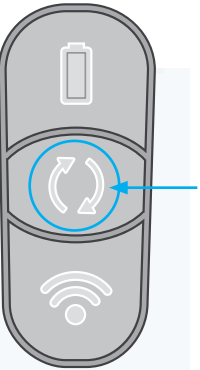
When **Flexion Lock** is engaged and the knee is unweighted, it will be locked in both **flexion** and **extension**.


Feature: Sitting Mode
Function: The Knee will flex with no resistance when the thigh reaches the horizontal plane of flexion or greater.
Use:
Engage: Unweight Knee and bring to the horizontal plane. The Knee will then drop without resistance.

TIP!

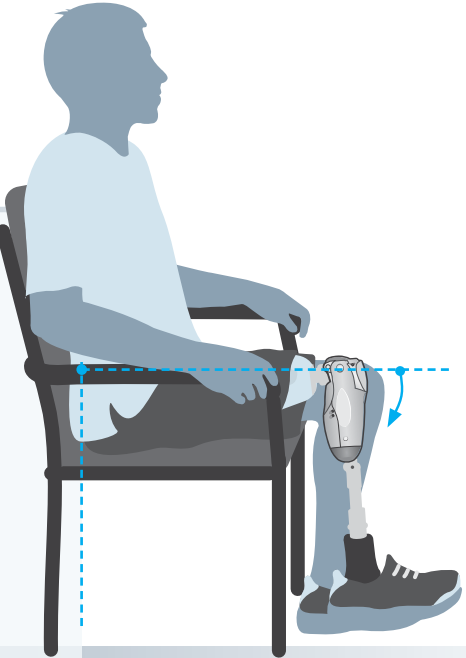When **Sitting Mode** is turned off, **Cycling** and **Obstacle Assist** will be disabled.

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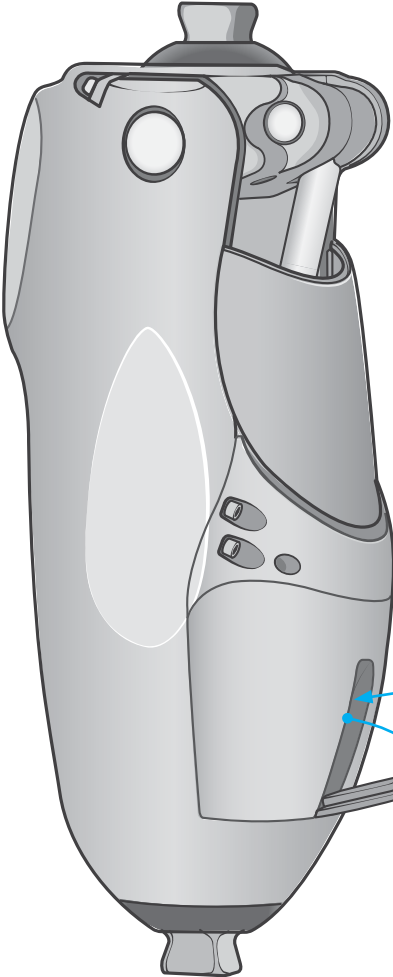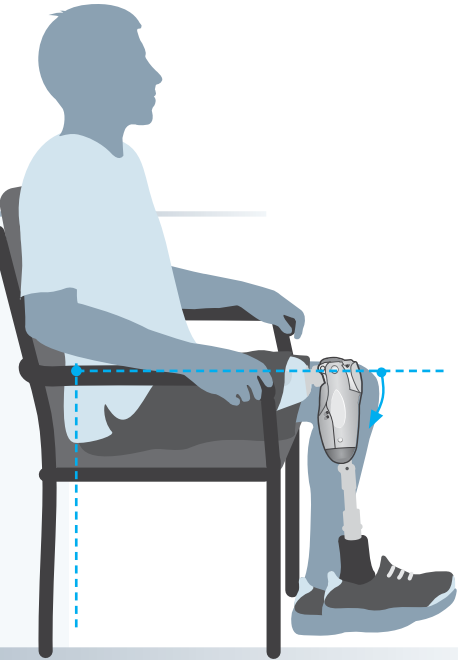
Keep clean of dust and debris:

- Use clean tap water to rinse off.

- A soft bristled brush (such as a toothbrush) is acceptable to dislodge stuck dirt on the outside of the knee only.

- Using any other chemicals or soaps may void the warranty.

- Dry thoroughly with a towel after rinsing.

- Do not use a hairdryer, which may damage the electronics and void the warranty.

QUATTRO is rated IP67, which is weatherproof, not waterproof:

- Safe for occasional submersion in fresh water up to 1m [3 ft.] for up to 30 minutes.

- Dry with a towel once out of fresh water.

- Should not be used for showering or bathing as soaps and gels will malfunction unit.

- Should not go in saltwater or chlorinated water as these may cause corrosion.

- If sprayed by or accidentally submerged in saltwater or chlorinated water, immediately rinse with fresh water and dry with a towel.

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PROTEOR QUATTRO is a registered trademark of PROTEOR
© 2025 by PROTEOR | All rights reserved.