PROTEOR
QUATTRO[®]
MICROPROCESSOR KNEE

PROTEOR QUATTRO BOOSTER ADAPTER

ACC0010

EN	Instructions for use	FI	Käyttöohjeet
FR	Notice d'utilisation	PL	Instrukcja użytkowania
DE	Gebrauchsanweisung	CS	Návod k použití
IT	Istruzioni per l'uso	SK	Návod na používanie
ES	Instrucciones de uso	HR	Upute za uporabu
PT	Instruções de utilização	RU	Инструкция по использованию
NL	Gebruikershandleiding	UK	Інструкція з використання
DA	Brugervejledning	JA	取扱説明書
NO	Brukerveiledning	ZH	使用说明
SV	Bruksanvisning	AR	تعليمات الاستخدام

PROTEOR USA, LLC
1236 West Southern Ave, Suite 101
Tempe, Arizona 85282
UNITED STATES
☎ +1.855.450.7300
info@proteor.com
us.proteor.com

PROTEOR SAS
6 rue de la Redoute
21850 Saint-Apollinaire
FRANCE
☎ +33(0)3 80 78 42 42
cs@proteor.com
fr.proteor.com

EN Instructions for use

System Components

Part Description	Part Number
PROTEOR QUATTRO Booster Adapter	ACC0010

Terms/Symbols Used in This Guide

	Type BF applied part
	Electronic Equipment Should not be discarded as unsorted waste - must be sent to separate collection facilities for recovery and recycling
	FCC mark
	Manufacturer
	Refer to instruction manual / booklet
	CE marking
	EU Authorized Representative
	Caution
	Connected Indicator
	Charging Indicator

Warnings /Safety Precautions

- In case of damaged packaging, check the integrity of the device.
- Failure to comply with safety precautions can result in the QUATTRO BOOSTER ADAPTER malfunctioning and risk of injury to the user.
- Do not attempt to open or modify any component(s) and DO NOT attempt to make repairs on the product. The QUATTRO BOOSTER ADAPTER contains no user-serviceable parts.
- If any issues or concerns with the product arise, contact your Prosthetist for assistance.
- Improper and unsafe use of the QUATTRO or the BOOSTER ADAPTER may result in injury.
- The QUATTRO BOOSTER ADAPTER should not be disposed of in residential or commercial garbage because it contains electronic components.

- The QUATTRO BOOSTER ADAPTER contains small parts and cords that can be potential choking hazards.
- If the QUATTRO BOOSTER ADAPTER has been stored outside its Operating Temperature Range, allow the adapter to acclimate to the operating environment for a minimum of 30 minutes before use.
- Failure to comply with the Instructions for Use will void the warranty.
- Do not use the QUATTRO BOOSTER ADAPTER while driving, operating machinery, or power tools. This may create an unsafe tangle hazard.
- Do not use the QUATTRO BOOSTER ADAPTER near water. While the QUATTRO knee itself is water resistant, the QUATTRO BOOSTER ADAPTER is not water resistant and may malfunction or break when exposed to water.
- There are no Applied Parts in the QUATTRO BOOSTER ADAPTER. All external surfaces of the QUATTRO BOOSTER ADAPTER, including the cords are Accessible Parts.

Product Description

The QUATTRO BOOSTER ADAPTER charges your QUATTRO knee to full charge from an external battery or other USB-C power source, allowing for extended usage away from wall power. The BOOSTER ADAPTER may be used to charge a knee when not in use, or to extend the battery life of the knee while being worn. A fully charged 10,000 mAh battery will provide at least 1 full charge to your QUATTRO knee.

Indications

The QUATTRO BOOSTER ADAPTER is indicated for use by QUATTRO users who want to charge when wall power is inconvenient, not available, or USB-C power is preferred. Activities may include, but are not limited to:

- Camping, hiking, or other outdoor activities
- Travel where the user may not have wall adapters for foreign outlets
- Other charging where a wall outlet is inconvenient or impossible

Contraindications

The QUATTRO BOOSTER ADAPTER has no contraindications for use other than those for the QUATTRO knee itself (Reference

QUATTRO Instructions for Use documentation).

Side Effects

There are no known negative side effects. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority.

Basic Operations

1. Ready the USB-C power source. If you use an external battery, ensure it is fully charged.
2. Connect BOOSTER ADAPTER to a power source. Plug the USB-C connector of the BOOSTER ADAPTER Cable into the power source USB-C port.
3. Connect BOOSTER ADAPTER to QUATTRO:
Plug the other end of the BOOSTER ADAPTER into the knee power input port in the same manner as you would the original wall charger. The Blue CHARGING light on the BOOSTER ADAPTER may flash intermittently while the knee prepares to begin charging. This will wake your knee up from storage mode, just like a wall charger would.
4. Wait for CHARGING light to turn solid blue. When the knee is charging, the CHARGING light on the BOOSTER ADAPTER will light solid Blue. This may take 5-10 seconds. The knee takes the same time to charge using the BOOSTER ADAPTER as it does from the regular wall charger. If the CHARGING light on the BOOSTER ADAPTER is not lit solid, the knee is not charging. This may be for the following reasons:
 - The knee is already fully charged *or*
 - The external battery is empty or off *or*
 - You are attempting to charge an empty or nearly empty external battery and charge the knee from this external battery at the same time *or*
 - You are attempting to use an incompatible power source (in this case, the CONNECTED light will not be lit either) *or*
 - If the LED still does not turn solid, please contact your Prosthetist.
5. (Optional) Verify charging is complete:
When your QUATTRO is done charging, the blue CHARGING light on the BOOSTER ADAPTER may turn off by itself. You may

verify that your QUATTRO is charged by following steps in the QUATTRO Instructions for Use.

Considerations for Cosmesis Users

For amputees who intend to use the product with a cosmesis, there are two options:

- **Standard Operation**
The Prosthetist may cut a hole in the cosmesis allowing the user to access the QUATTRO charge port. In this case the QUATTRO wall charger and BOOSTER ADAPTER may be used as if there was no cosmesis.
 - **Pass-through Operation**
The Prosthetist may, at their judgement and the request of the user, use the BOOSTER ADAPTER as a charge port extension cable so that the cosmesis can be used with no visible external holes. If this is done, the user will charge exclusively using an external USB-C power source. The main charge port of the knee will no longer be accessible.
 1. The Prosthetist shall attach the charge plug of the BOOSTER ADAPTER to the charge port on the knee using an appropriate method, typically strong tape. **WARNING:** Liquid adhesives may leak between the charging pins and cause the knee to not charge.
 2. The Prosthetist shall run the BOOSTER ADAPTER up along the user's socket, anchoring it at specific points to the socket so that it is not pinched at any point during full range of motion, but leaving just enough slack for free movement. Ensure that any extra cable length is looped and secured as appropriate.
 3. The Prosthetist shall anchor the main body of the BOOSTER ADAPTER in a visible location, so that the user can see the lights on the BOOSTER ADAPTER.
-  Avoidance of cable pinching is the responsibility of the Prosthetist. If the Prosthetist does not believe it is possible to prevent cable pinching during full joint range of motion, do not install the BOOSTER ADAPTER in this manner and use Standard Operation instead. If the cable is damaged due to being installed for pass-through operation, the knee may not be able to charge. PROTEOR is not responsible for units damaged by installation as pass-through chargers.

Environmental

- Operating Temperature Range: 0 to 40 °C [32 to 113 °F]
- Storage Temperature Range: -20 to 60 °C [-13 to 140 °F]
- Altitude Range: 0-5,000 m [0-16,400 ft.]
- Storage and operating relative humidity range: 0% to 100%, not including condensation
- Water resistance: Not rated.

⚠ Do not expose the QUATTRO BOOSTER ADAPTER to water.

Cleaning

A dry cloth or alcohol wipes may be used to clean the outside surface of the QUATTRO BOOSTER ADAPTER.

Warranty and Service

The BOOSTER ADAPTER is provided with a 12-month warranty covering all manufacturer defects only if the product is used according to the manufacturer's recommendations.

Technical Support

- For User Support, please contact the Certified Prosthetist.
- For Prosthetist technical support, or to schedule service, please contact PROTEOR.

Electromagnetic Compatibility Precautions

The PROTEOR QUATTRO needs special precautions regarding electromagnetic compatibility (EMC). Specifically, it needs to be installed and put into service according to the EMC information provided as follows:

- The QUATTRO may be susceptible to electromagnetic interference from portable and mobile RF communications devices such as mobile (cellular) telephones.
- The QUATTRO may be interfered with by other equipment, even if that other equipment complies with CISPR EMISSION requirements.

Guidance and Manufacturer's Declaration – Electromagnetic Emissions

The QUATTRO BOOSTER ADAPTER is intended for use in the electromagnetic

environment specified below. The customer or user of QUATTRO should ensure that it is used in such an environment.

Emission test	Compliance	Electromagnetic Environment Guidance
RF Emissions EN 55011 / CISPR	Group 1	QUATTRO Booster Adapter uses RF energy only for its internal function. Therefore, its RF emissions are low and are not likely to cause any interference in nearby electronic equipment.
RF Emissions EN 55011 / CISPR	Class B	QUATTRO Booster Adapter is suitable for use in all establishments, including domestic establishments and those directly connected to the public low voltage power supply network that supplies building used for domestic purposes.

Regulatory compliance

The QUATTRO BOOSTER ADAPTER is a CE-marked medical device and is certified as conforming with Regulation (EU) 2017/745, EN 55011 Class B, IEC 60601-1 and IEC 60601-1-2. The device complies with Part 18 of the FCC Rules.

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