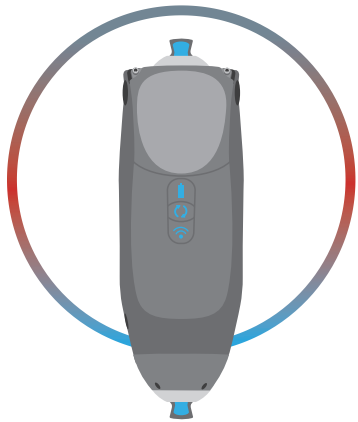
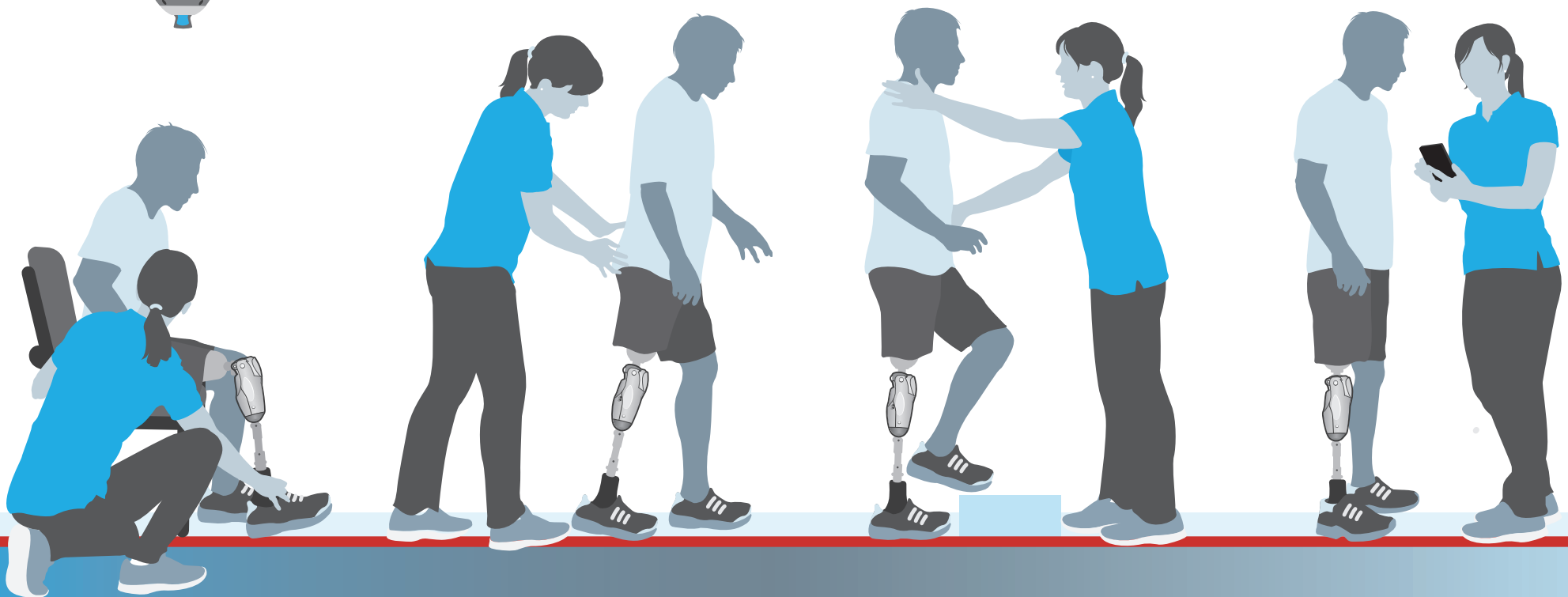
- ▶ CLINICIAN FULL SETUP GUIDE
- ▶ GUIDE D'INSTALLATION COMPLET DU CLINICIEN
- ▶ GUÍA DE CONFIGURACIÓN COMPLETA PARA EL CLÍNICO
- ▶ SETUP GUIDE FÜR ORTHOPÄDIETECHNIKER

PROTEOR

QUATTRO 2

MICROPROCESSOR KNEE

▶ GaitLab App

▶ Application
GaitLab

▶ Aplicación
GaitLab

▶ GaitLab-App

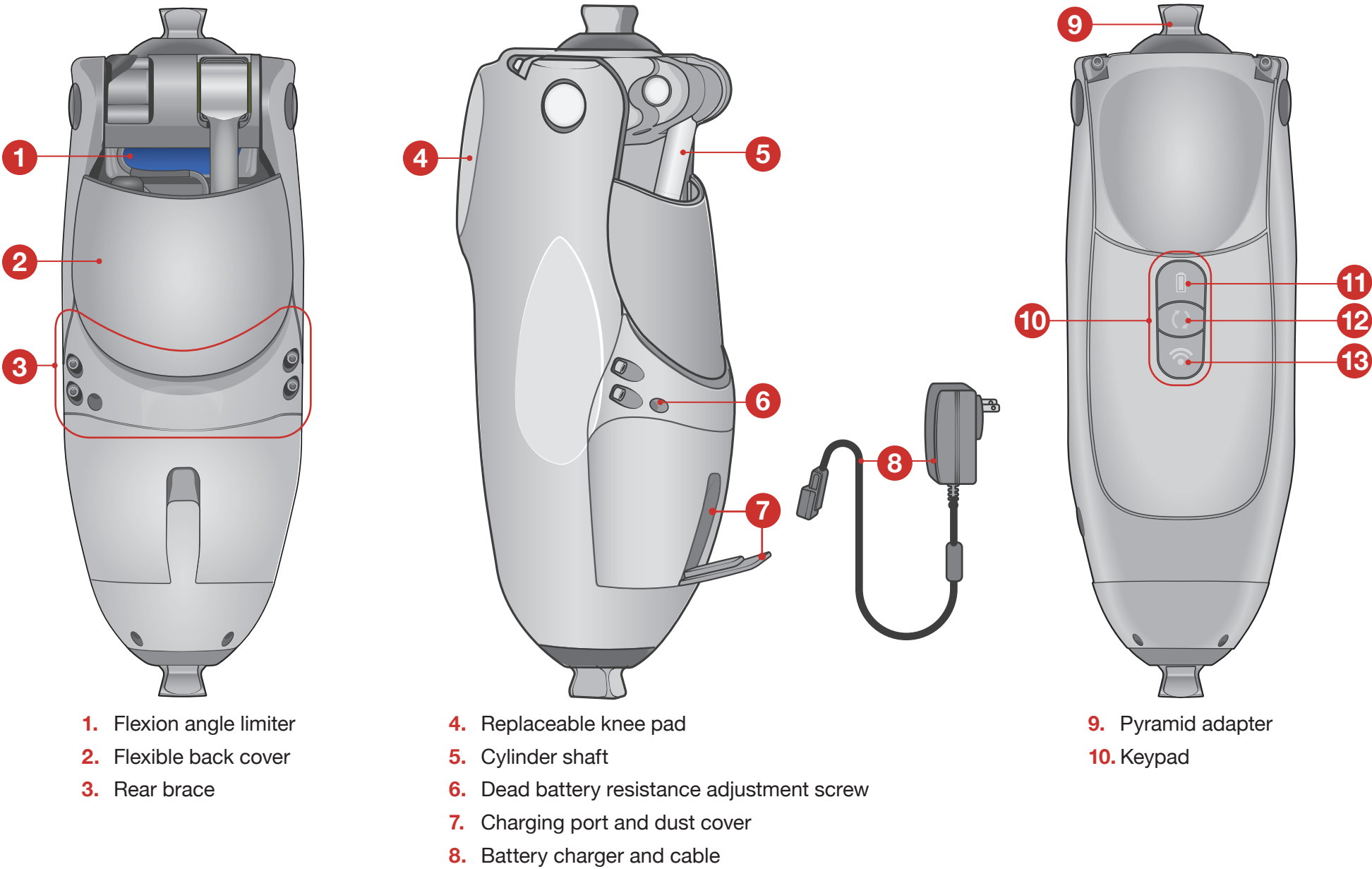
Launching worldwide 1-September, 2026

1

PROTEOR QUATTRO 2

Microprocessor Knee


Button	Status	Charge level	LED description
11 Battery Status	Fully charged	51–100%	
	Charged	21%–50%	
	Low	11–20%	
	Very low	1–10%	
	Knee reboot	Hold 6 seconds	
Button	Function	Status	LED description
12 Change Mode	Check activity Press and release	Unlocked	Walking: Alternate:
		Locked	
	Enable/disable lock Press twice	Lock engaged	
		Locked engaged, with very low battery	
		Locked disengaged	Previous LED activity resumes
	Toggle between Walking and Alternate Long press	Current activity: Walking	
		Current activity: Alternate	
		Current activity: Walking Alternate activity not enabled in patient app or unsafe to switch activities	
Button	Function	Description	LED description
13 Wireless	Pair with knee	Pairs with devices	None

LED Key	
Solid green	
Blinking green	
Solid red	
Blinking red	
Blinking blue	
Solid white	
Slow blinking white	
Fast blinking white	

2

GET STARTED

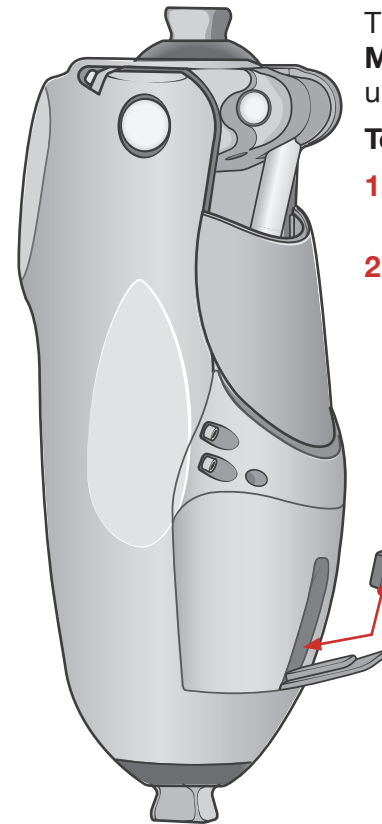
Wake the knee

The QUATTRO knee ships in **Storage Mode**. The knee will not perform (or wake) until it is plugged into a power supply.

To wake the knee:

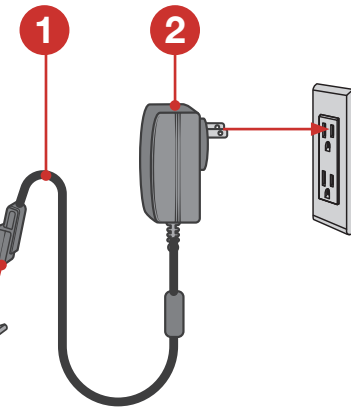
1. Insert the cable or charged booster battery pack into the charger port.
2. Insert the charger into a power source.

IMPORTANT!



- Do not leave the knee unattended. The battery will discharge when idle.
- To save battery life, place the knee in **Storage Mode** by pressing and holding both the **Change Mode** and **Wireless** buttons until they flash red, white, and blue
- Ensure you have the proper power source to wake the knee before putting the knee into **Storage Mode**.

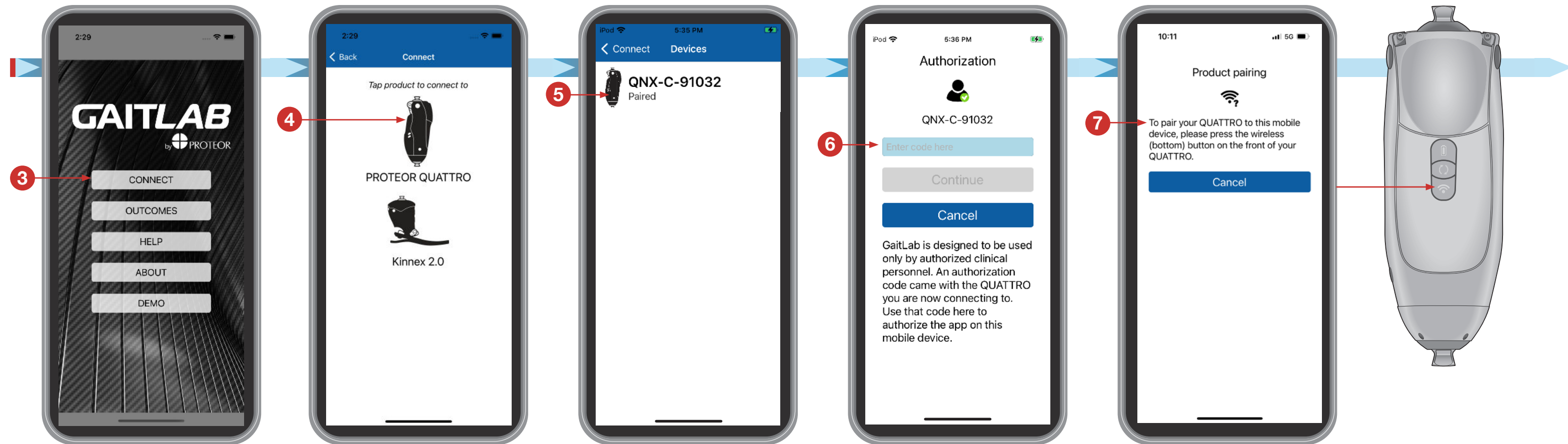Connect




3 QUATTRO 2 FULL SETUP

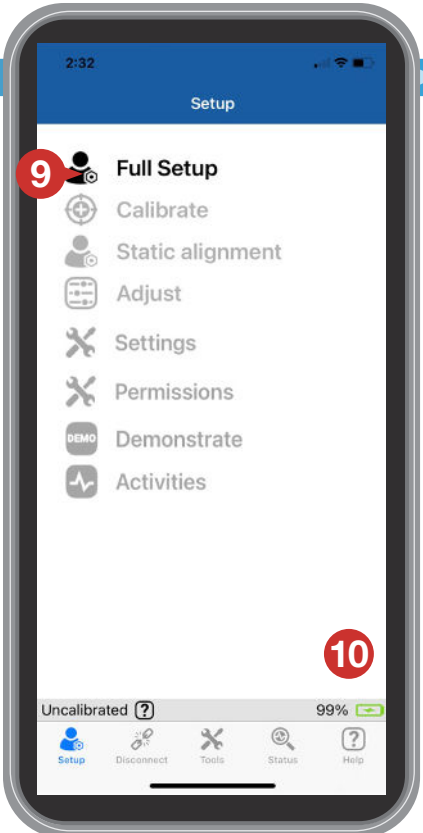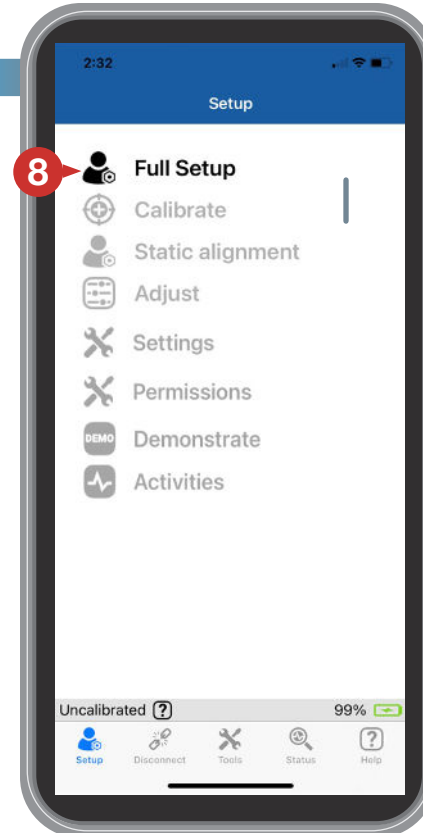Setup


Alignment

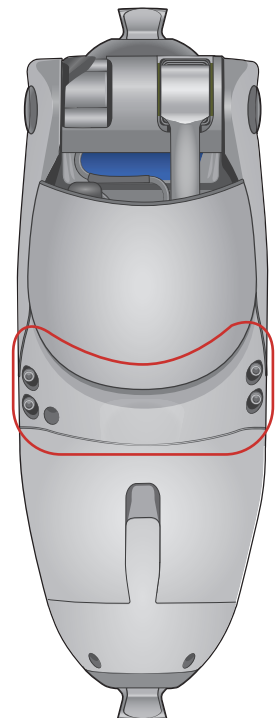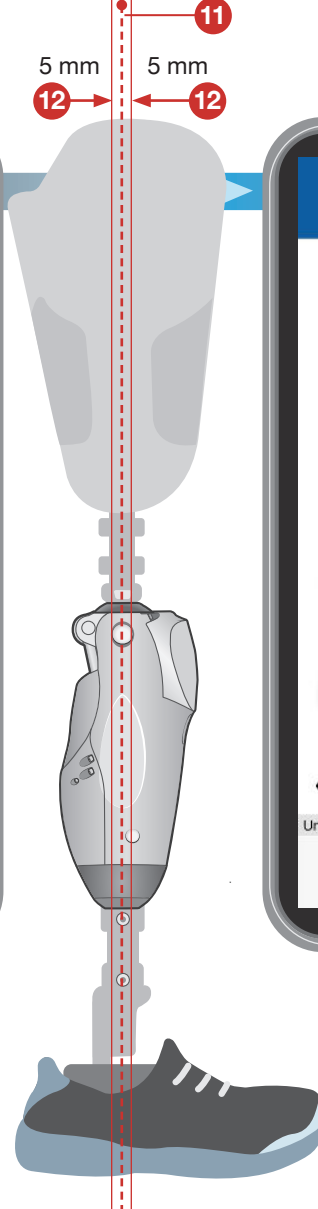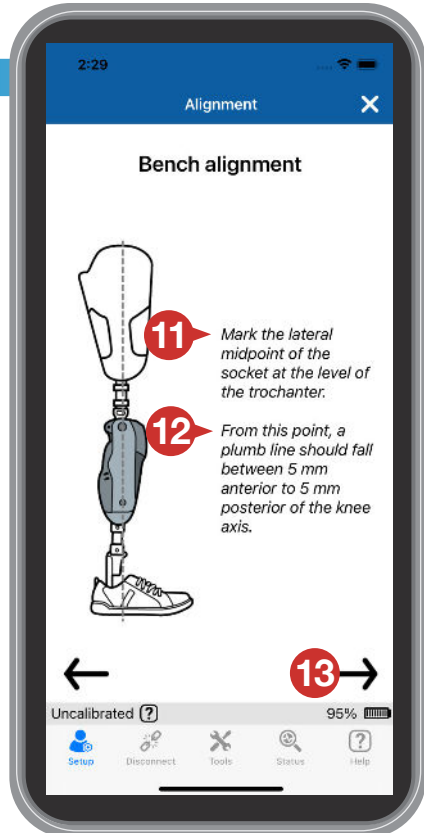
Calibration

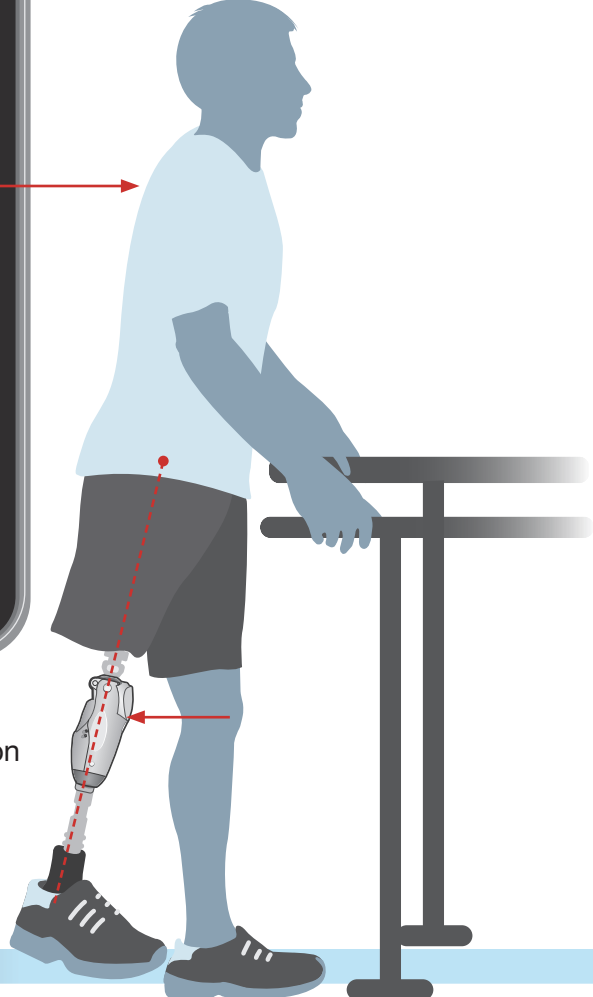Press Ready
after the patient is in position

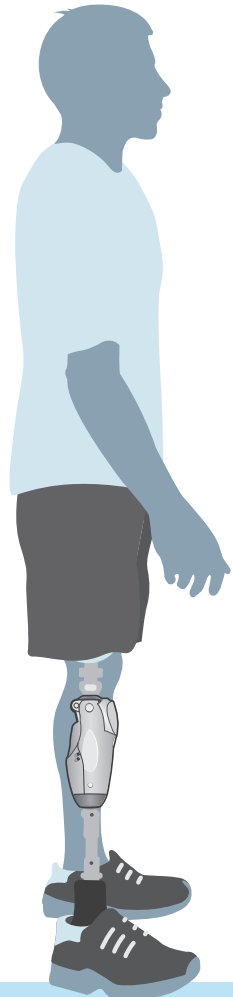Press Ready
after the patient is in position

Alignment

2:29

Calibration

Walking

The knee will unlock after pressing 'Ready'

Find a place where the patient can safely take 10+ consecutive steps, using a walker for stability if needed.

Ask the patient to walk at a self-selected speed after pressing 'Ready'.

Ready

18

Uncalibrated ? 95%

Setup Disconnect Tools Status Help

2:29

Calibration

Walking

2 steps detected

Uncalibrated ? 95%

Setup Disconnect Tools Status Help

2:29

Alignment

Static alignment

199

19 Stance Flexion: 0-10 mm posterior to knee center

20 No Stance Flexion: 0-10 mm anterior to knee center

21

Walking 95%

Setup Disconnect Tools Status Help

0-10 mm

0-10 mm

19

20

Press **Ready** before the patient begins to take steps

Adjust

2:29

Adjust

Sitting Resistance

50

22

Heel Rise Resistance

50

Walking 95%

Setup Disconnect Tools Status Help

Provides flexion resistance while **sitting**.

When delayed due to high resistance

When rapid due to low resistance

2:29

Adjust

Sitting Resistance

50

23

Heel Rise Resistance

50

Walking 95%

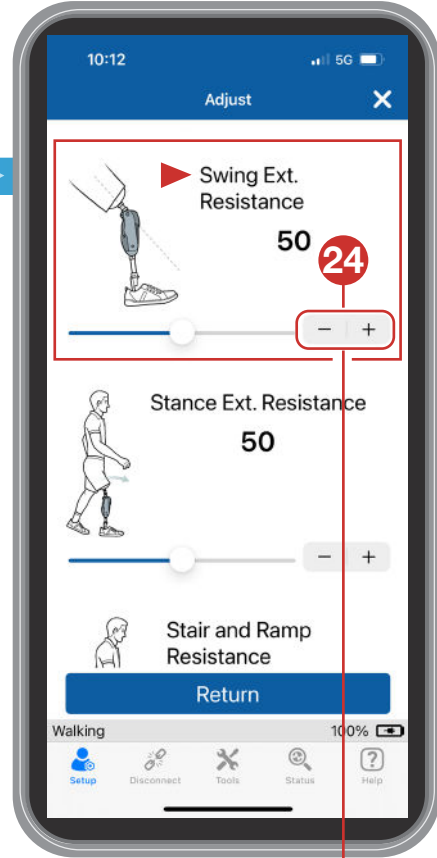
Setup Disconnect Tools Status Help

Controls **heel** rise resistance.

TIP! Scroll down for additional adjustments.

Increases heel rise


Decreases heel rise


Controls **swing extension resistance**.

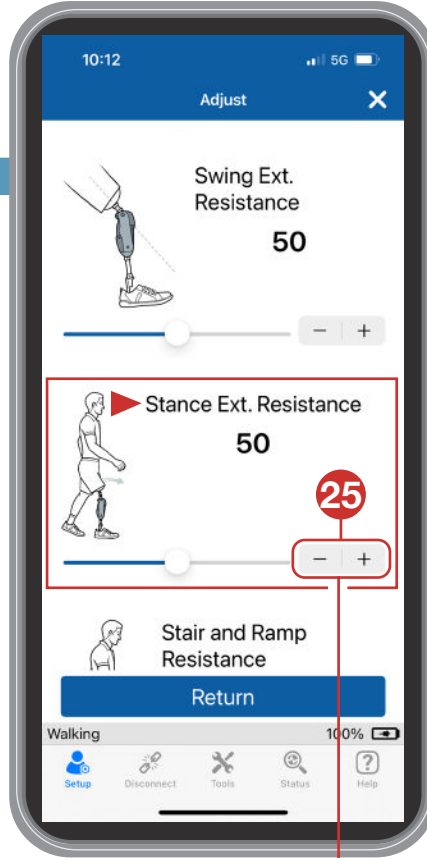- To decrease resistance
- + To increase resistance

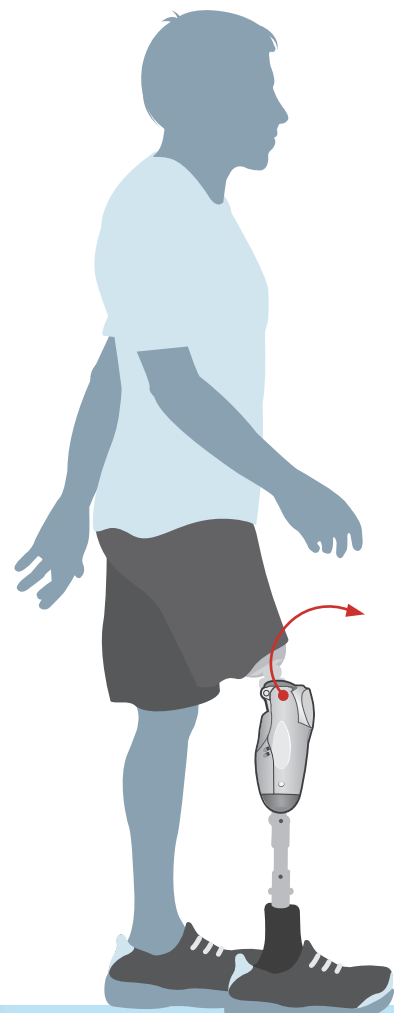
Controls **stance extension resistance**.


- To decrease resistance
- + To increase resistance

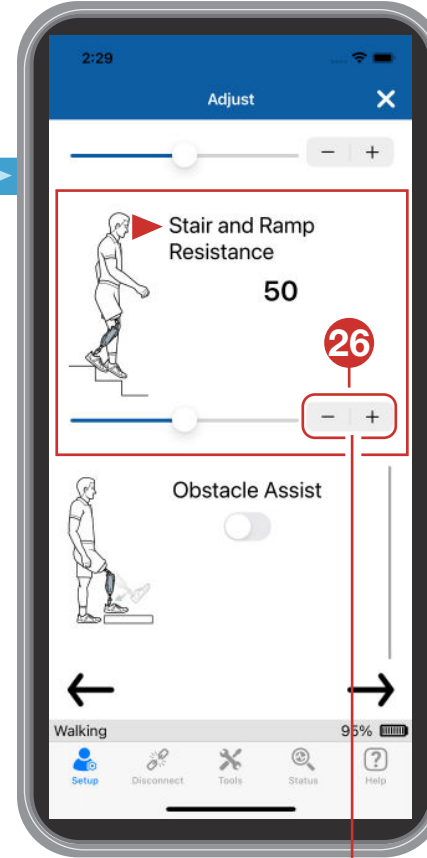
NOTE
Only adjusted when walking with **Stance Flexion**.

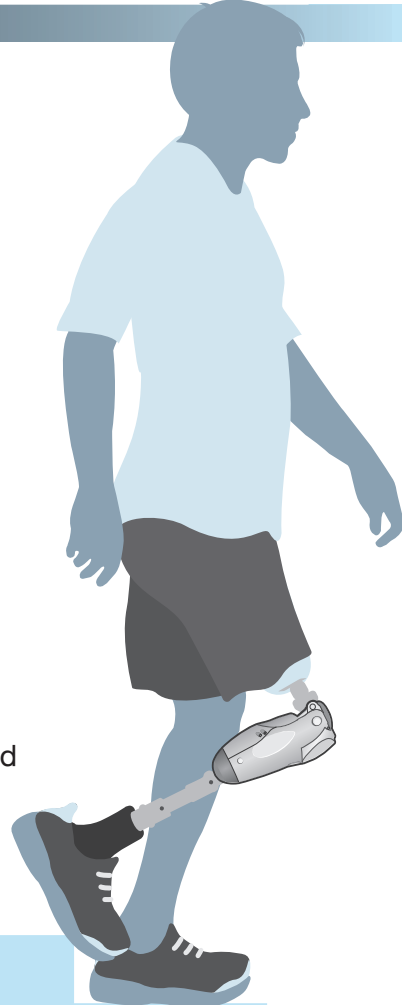
Provides independent resistance for **stair and ramp decent**.

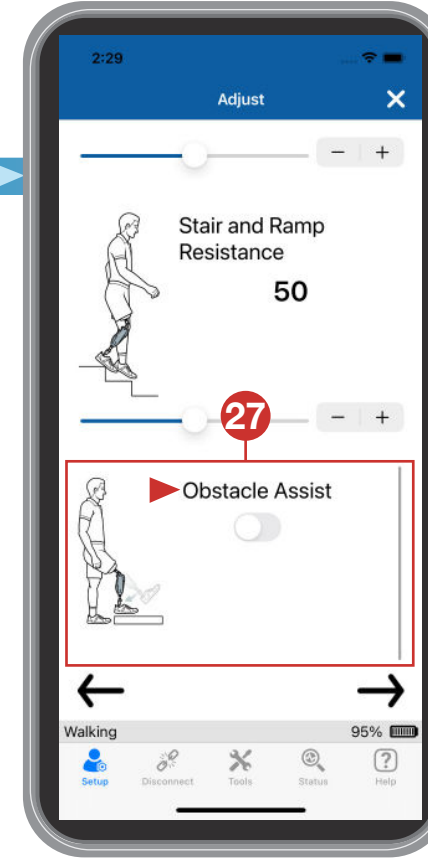- When knee flexion is delayed due to high resistance
- + When knee flexion is rapid due to low resistance

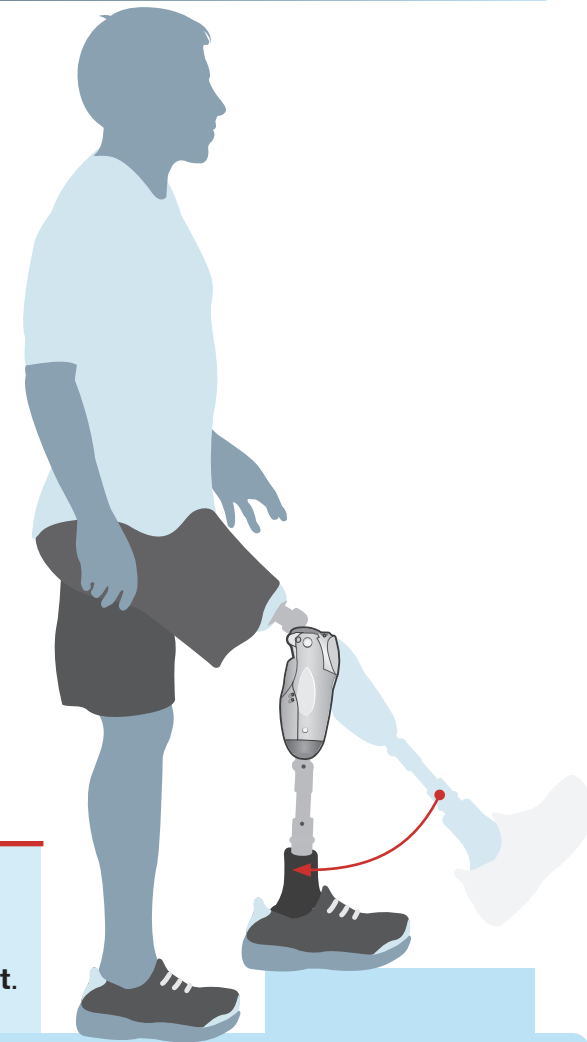
Provides help encountering an **obstacle**.


NOTE
If **Sitting Mode** is turned off, it will also disable **Obstacle Assist**.

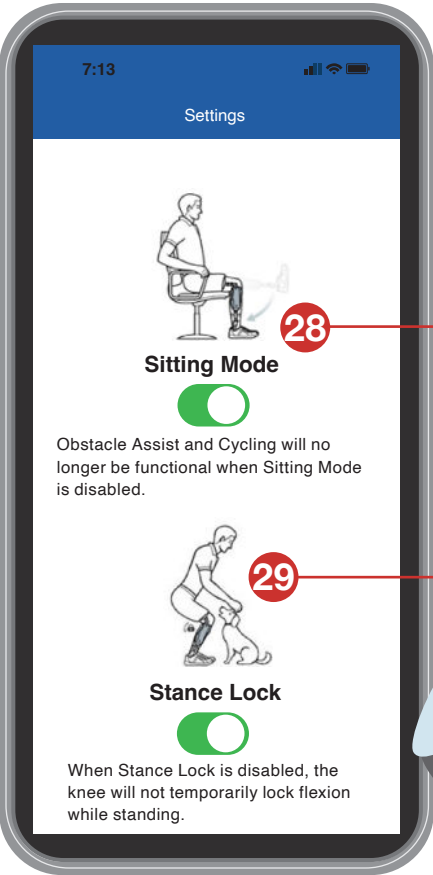Settings


TIP!

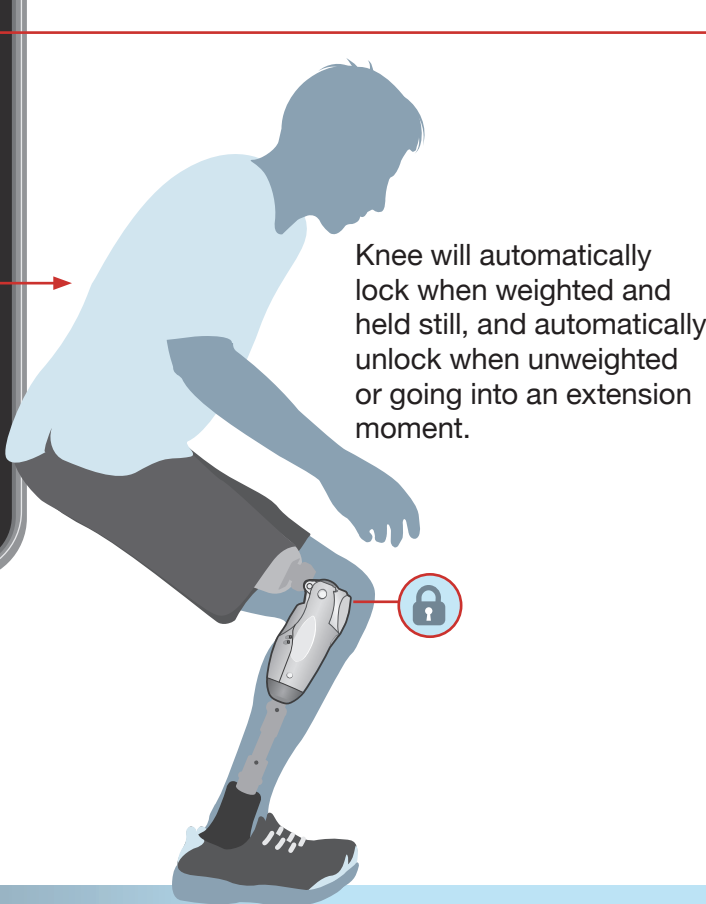
Scroll down for additional adjustments.

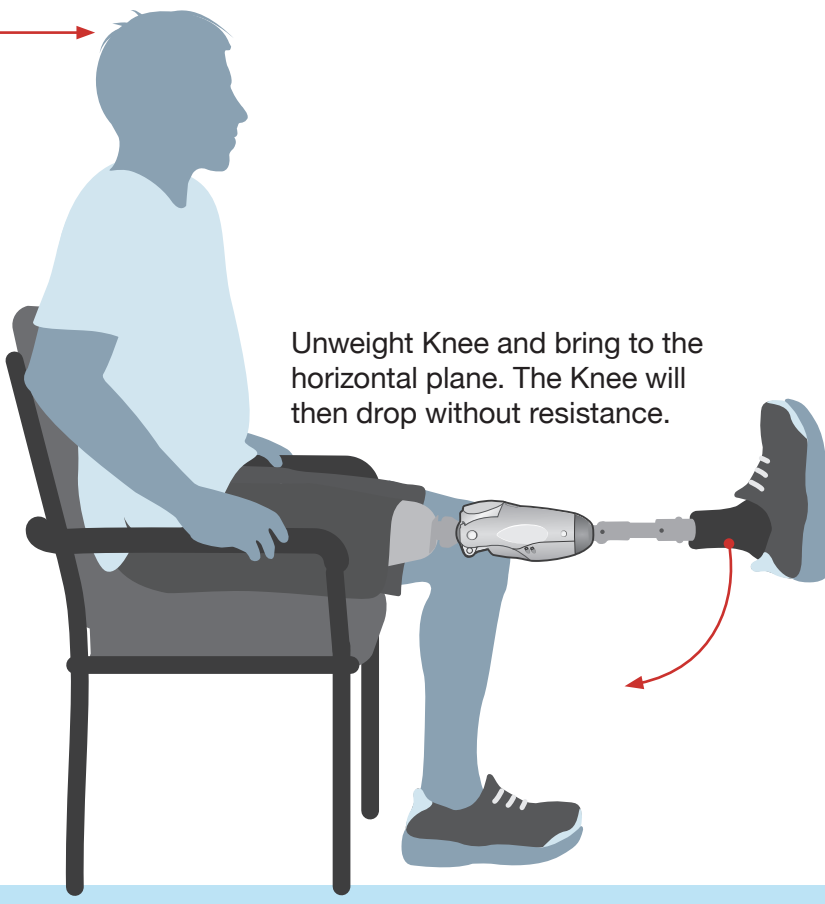Knee will automatically lock when weighted and held still, and automatically unlock when unweighted or going into an extension moment.

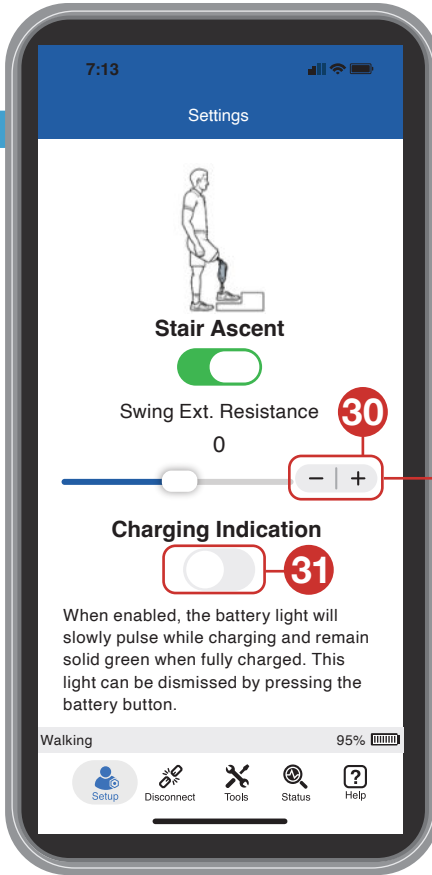Unweight Knee and bring to the horizontal plane. The Knee will then drop without resistance.


NOTE

Does not affect Walking Swing Extension Resistance.


When toe lands on step
When heel lands on step

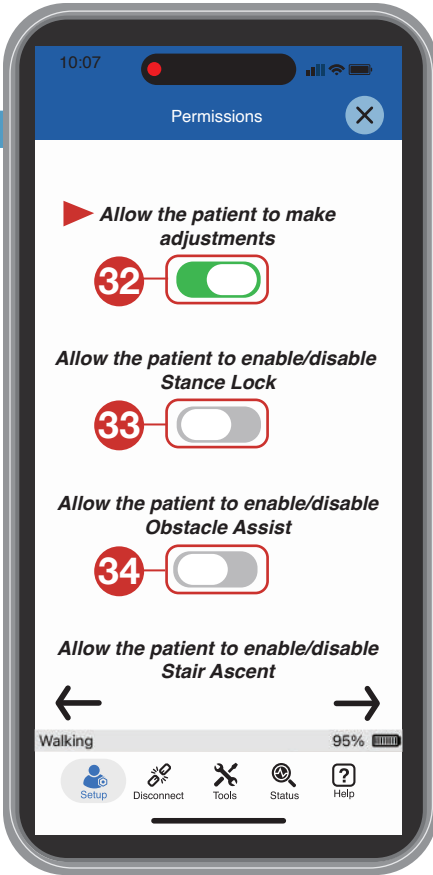Permissions

NOTE

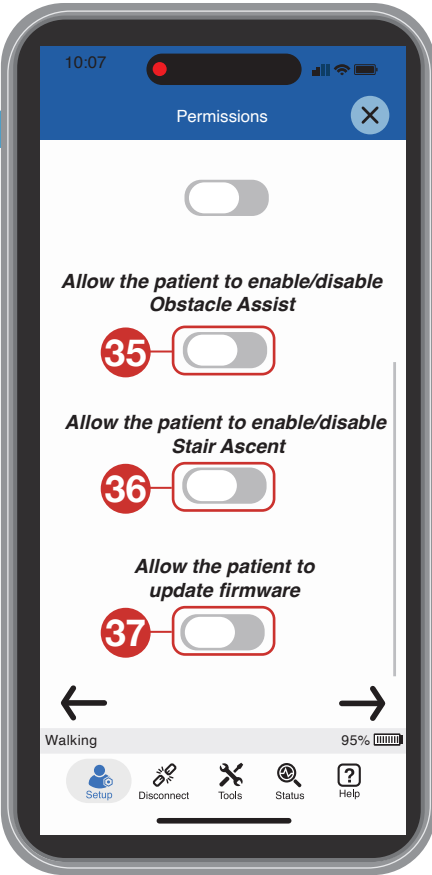If permissions are not enabled, these features will not be seen in the patient app.

Demonstrate

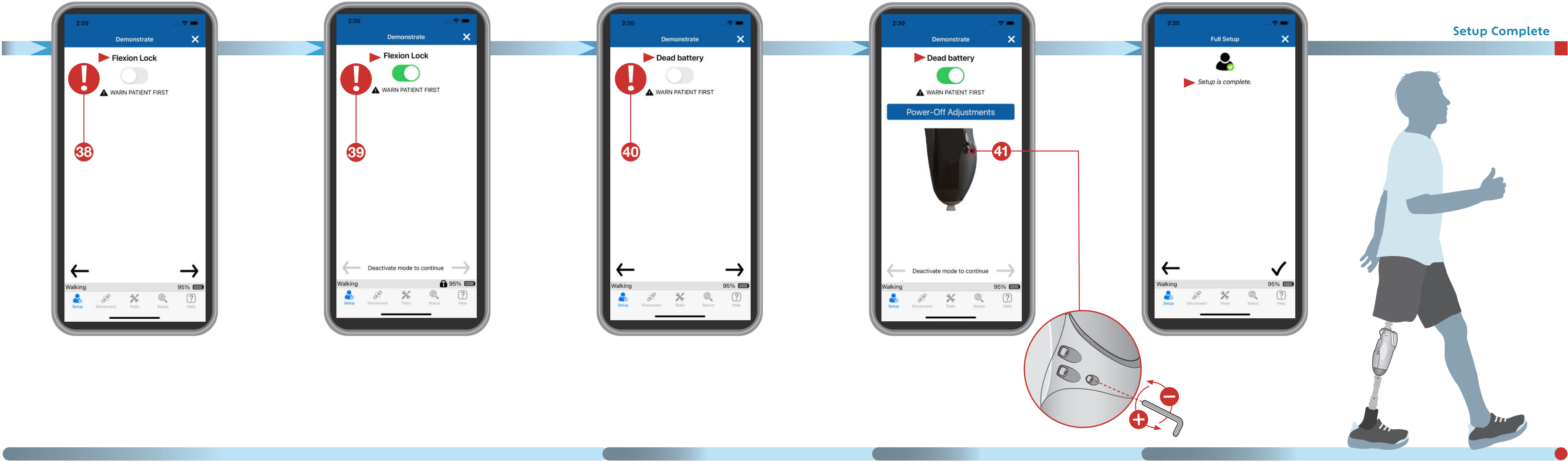
When the **Flexion Lock** is engaged, it blocks flexion but allows ratcheting up to full extension.

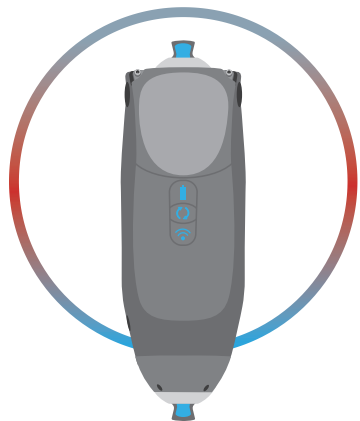
Demonstrate and Adjust

Perform resistance adjustments using a fully discharged battery.


Full Setup

Setup Complete

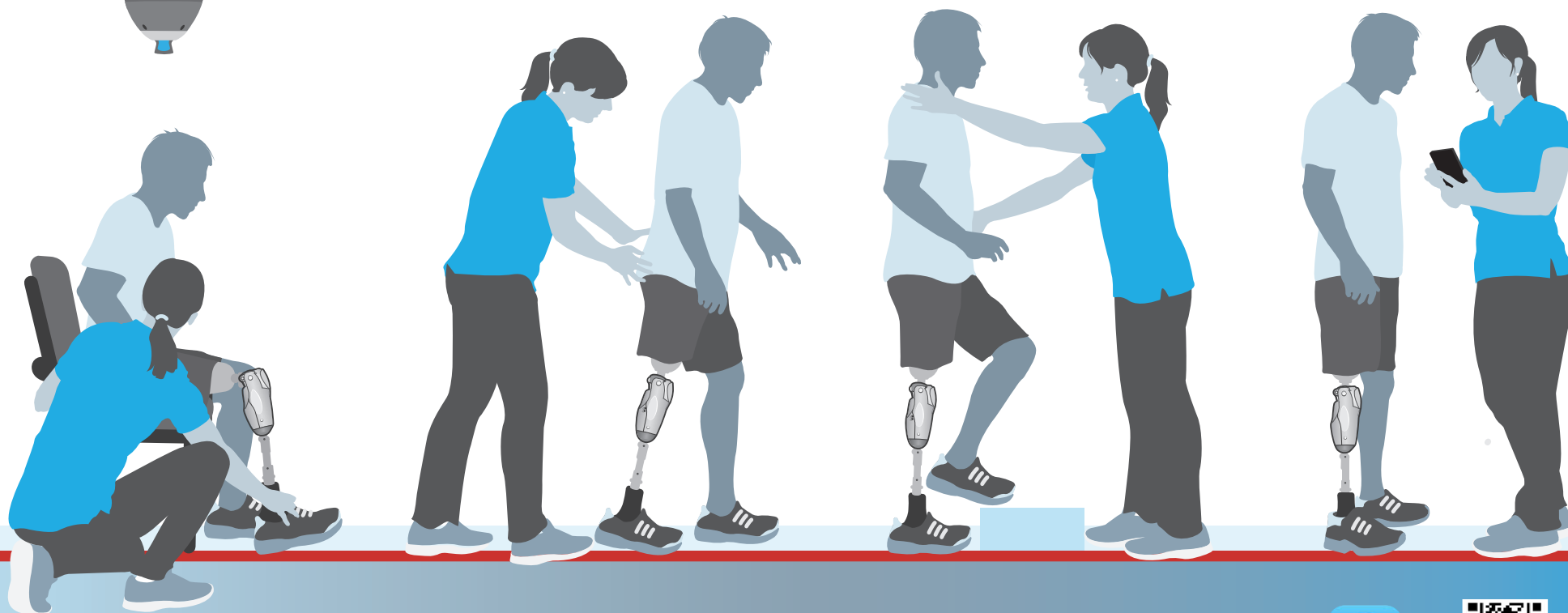- ▶ CLINICIAN FULL SETUP GUIDE
- ▶ GUIDE D'INSTALLATION COMPLET DU CLINICIEN
- ▶ GUÍA DE CONFIGURACIÓN COMPLETA PARA EL CLÍNICO
- ▶ SETUP GUIDE FÜR ORTHOPÄDIETECHNIKER


▶ GaitLab App

▶ Application
GaitLab

▶ Aplicación
GaitLab

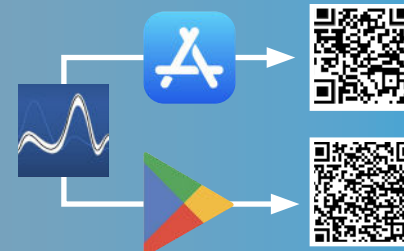
▶ GaitLab-App


1

PROTEOR QUATTRO[®]

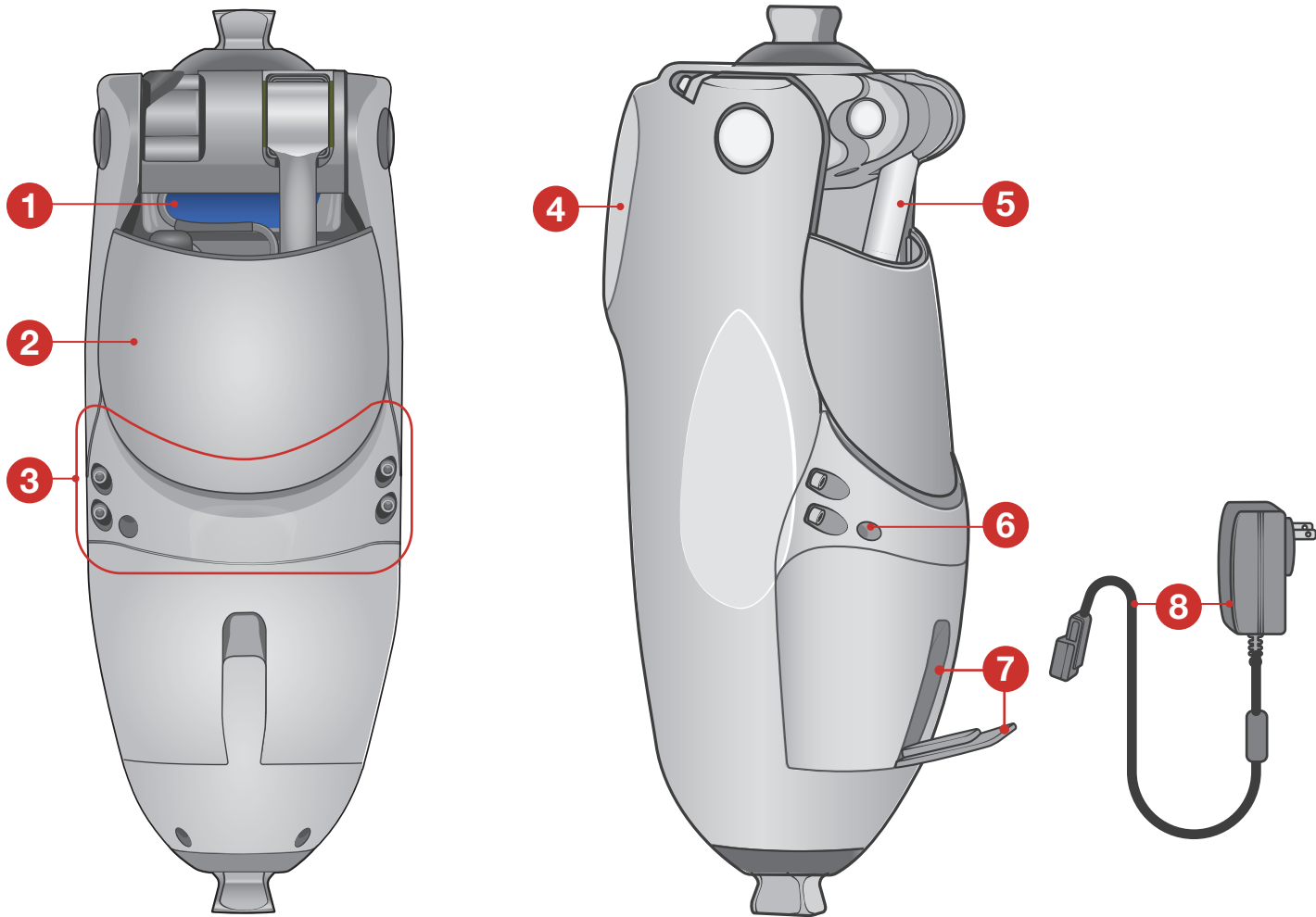Microprocessor Knee

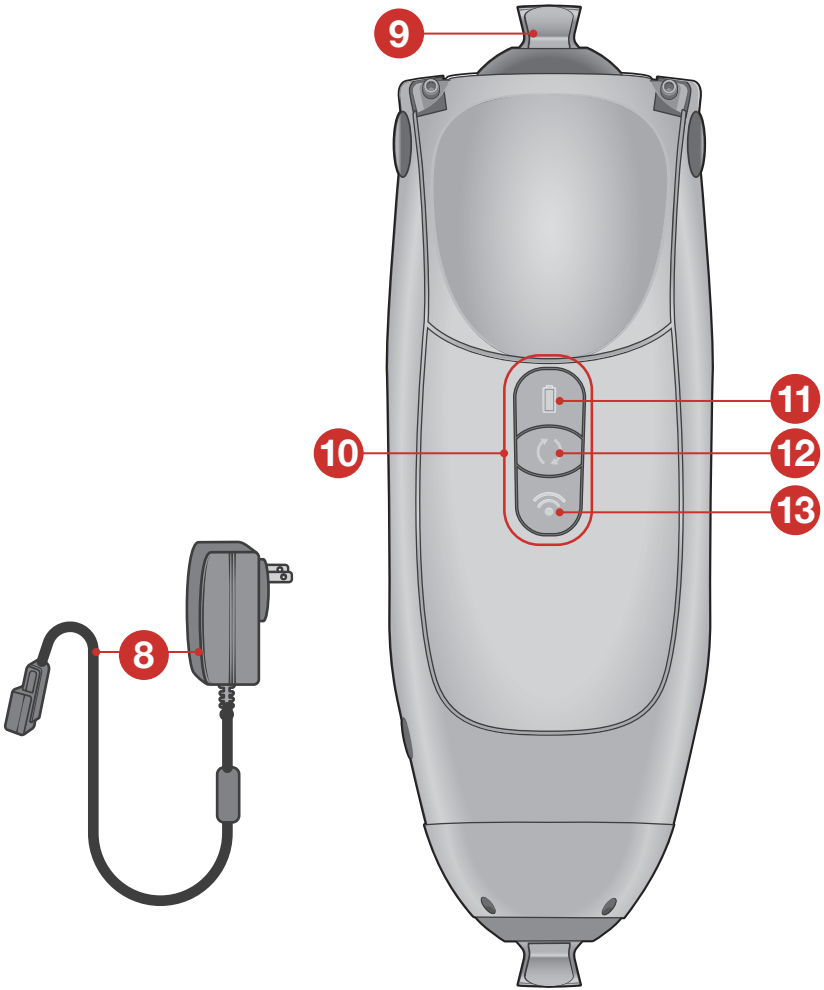1. Flexion angle limiter
2. Flexible back cover
3. Rear brace
4. Replaceable knee pad
5. Cylinder shaft
6. Dead battery resistance adjustment screw
7. Charging port and dust cover
8. Battery charger and cable

9. Pyramid adapter
10. Keypad


Button	Status	Charge level	LED description
11 Battery Status	Fully charged	51–100%	
	Charged	21%–50%	
	Low	11–20%	
	Very low	1–10%	
	Knee reboot	Hold 6 seconds	
Button	Function	Status	LED description
12 Change Mode	Check activity Press and release	Unlocked	Walking: Alternate:
		Locked	
	Enable/disable lock Press twice	Lock engaged	
		Locked engaged, with very low battery	
		Locked disengaged	Previous LED activity resumes
	Toggle between Walking and Alternate Long press	Current activity: Walking	
		Current activity: Alternate	
		Current activity: Walking Alternate activity not enabled in patient app or unsafe to switch activities	
Button	Function	Description	LED description
13 Wireless	Pair with knee	Pairs with devices	None

LED Key	
Solid green	
Blinking green	
Solid red	
Blinking red	
Blinking blue	
Solid white	
Slow blinking white	
Fast blinking white	

2

GET STARTED

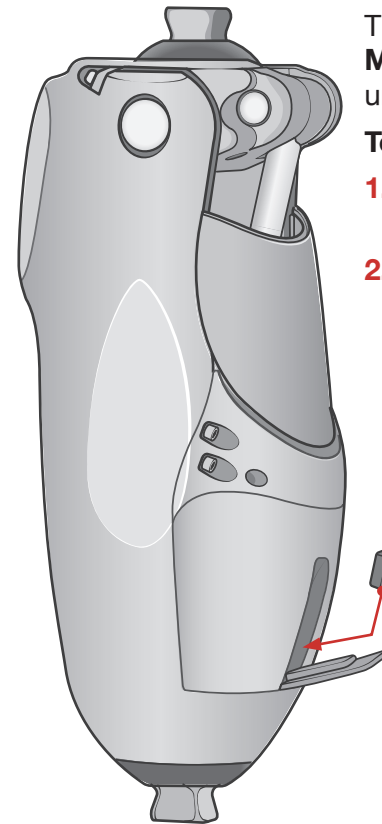
Wake the knee

The QUATTRO knee ships in **Storage Mode**. The knee will not perform (or wake) until it is plugged into a power supply.

To wake the knee:





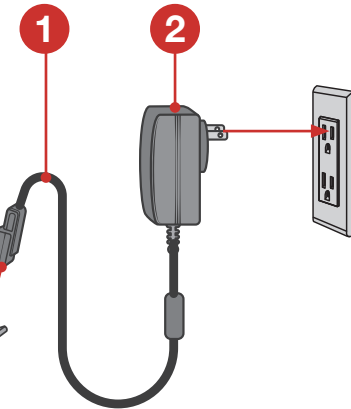
1. Insert the cable or charged booster battery pack into the charger port.
2. Insert the charger into a power source.

IMPORTANT!


- Do not leave the knee unattended. The battery will discharge when idle.
- To save battery life, place the knee in **Storage Mode** by pressing and holding both the **Change Mode** and **Wireless** buttons until they flash red, white, and blue
- Ensure you have the proper power source to wake the knee before putting the knee into **Storage Mode**.

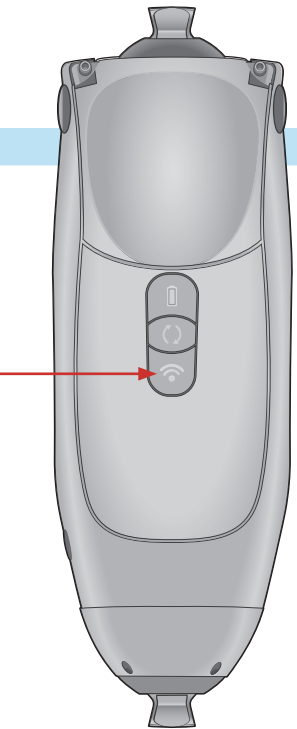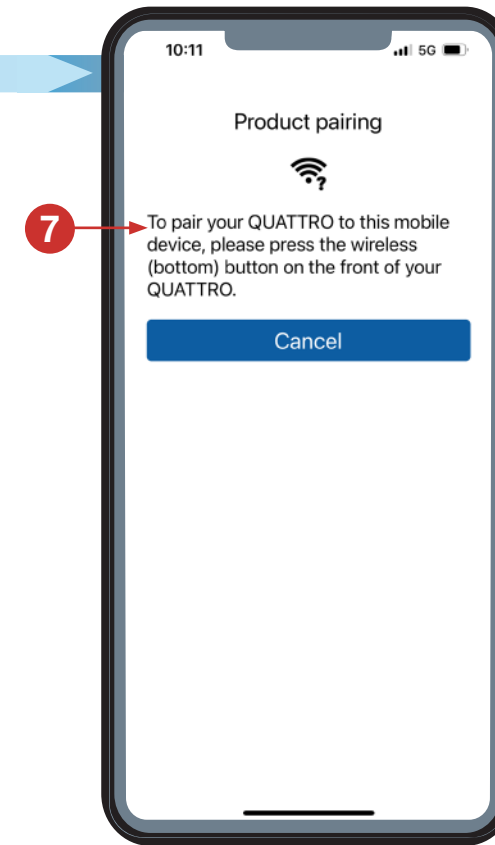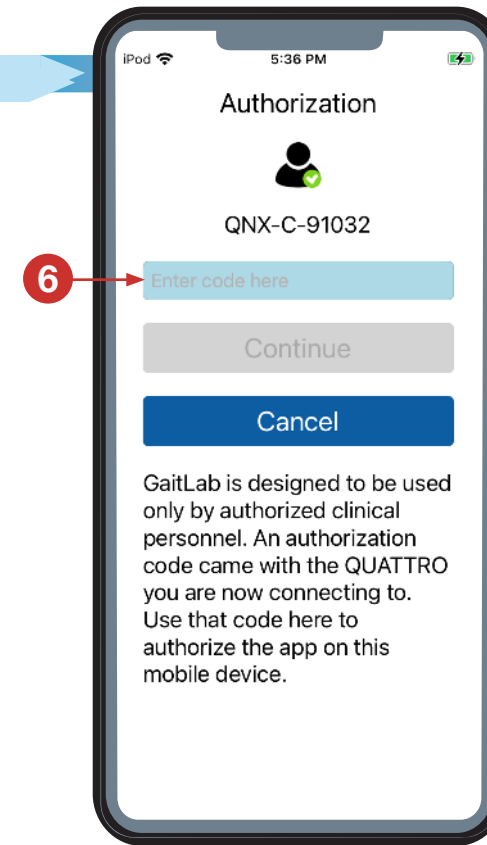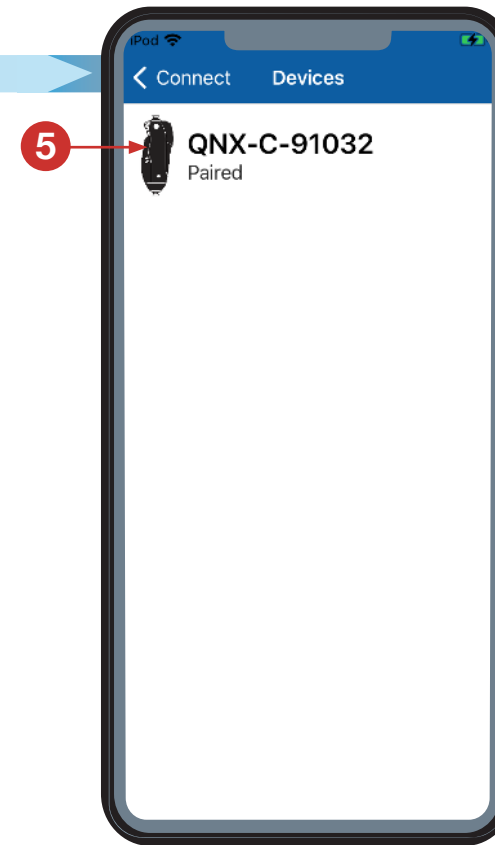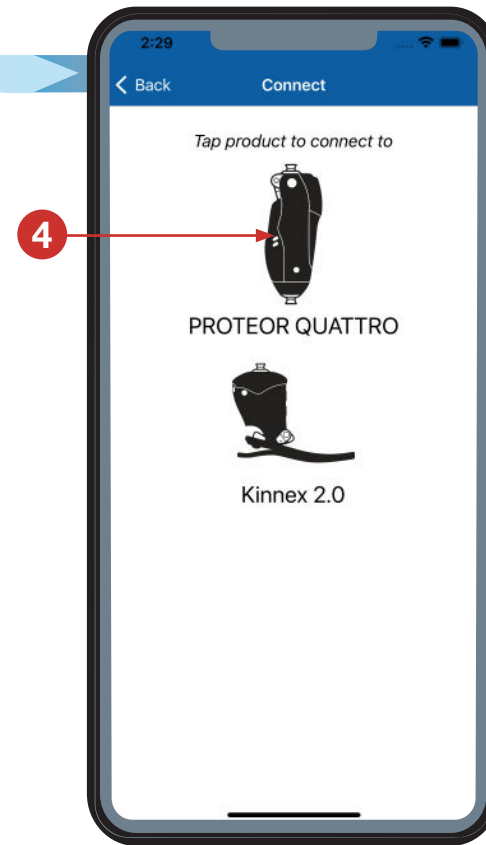Connect





3 QUATTRO FULL SETUP

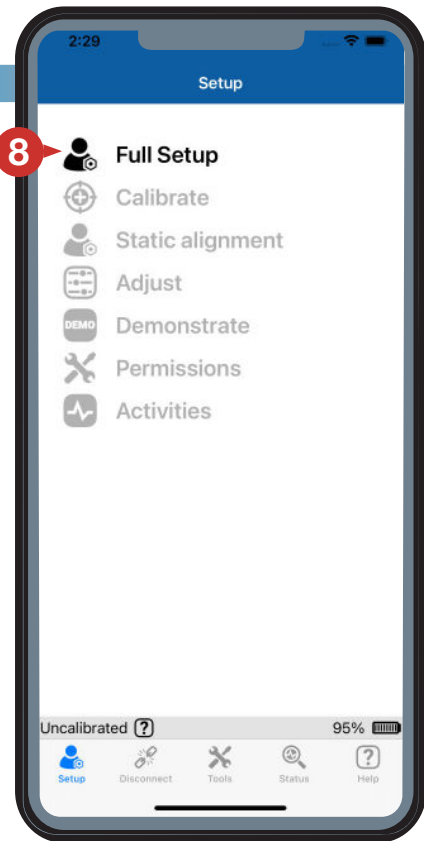
Setup



Alignment

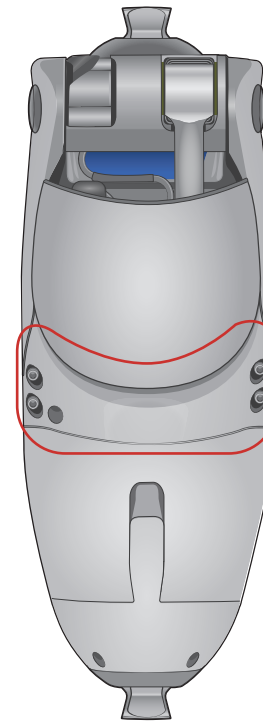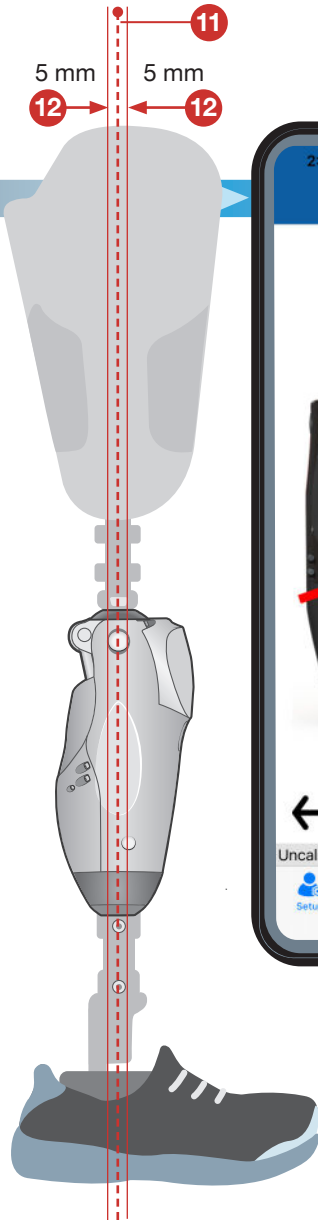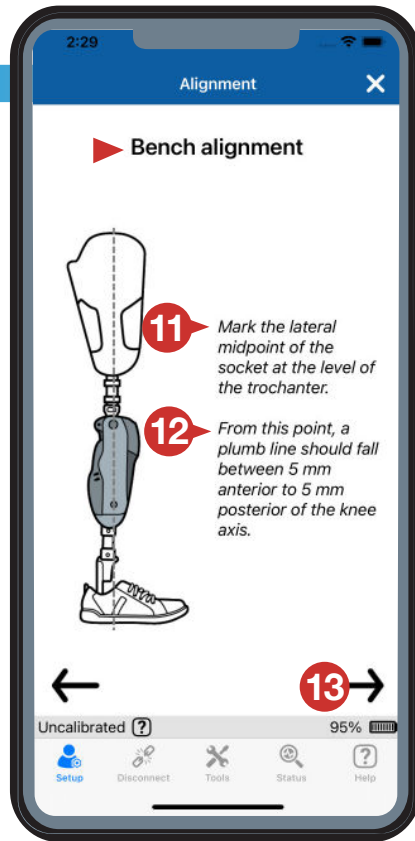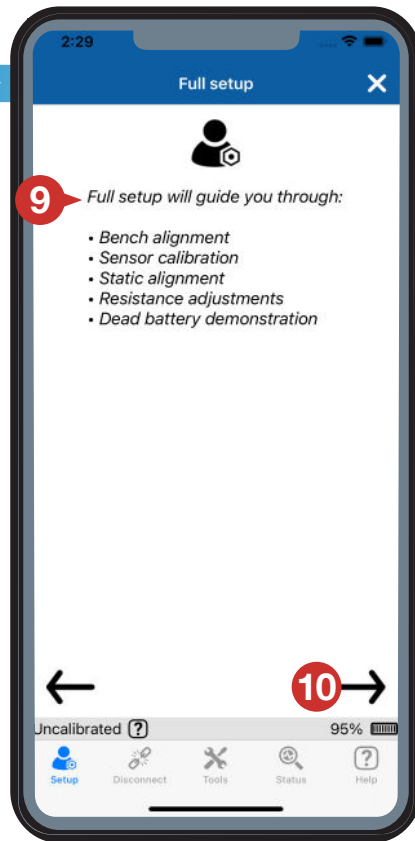Calibration

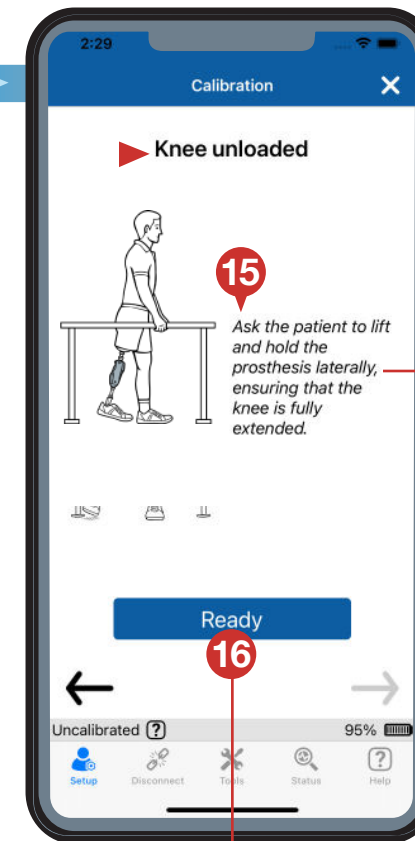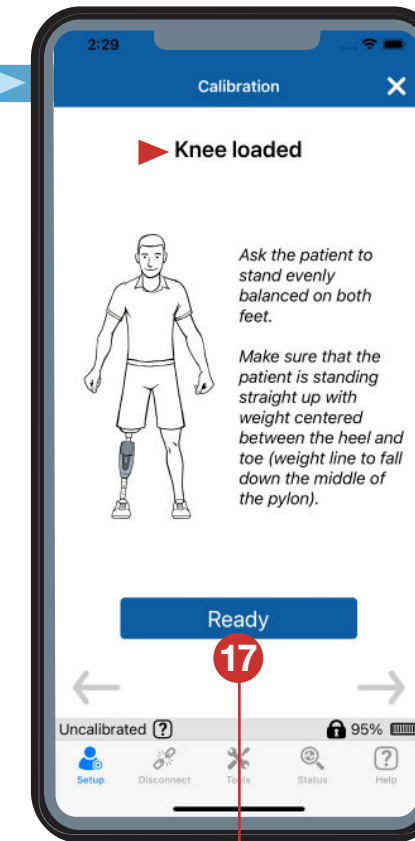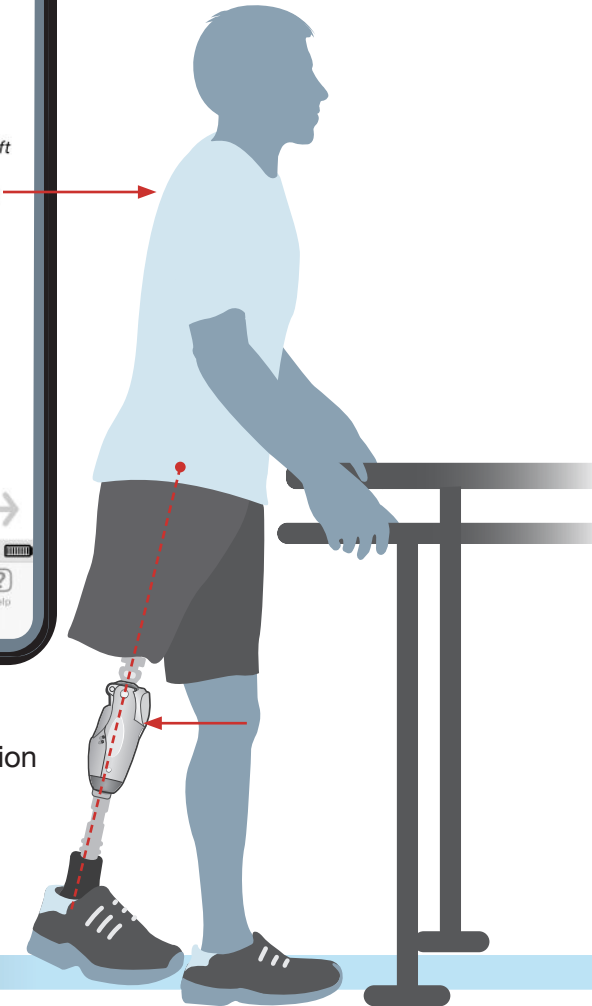
Press **Ready**
after the patient is in position


Press **Ready**
after the patient is in position

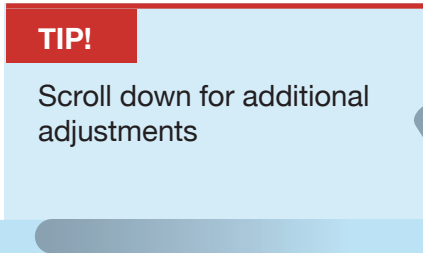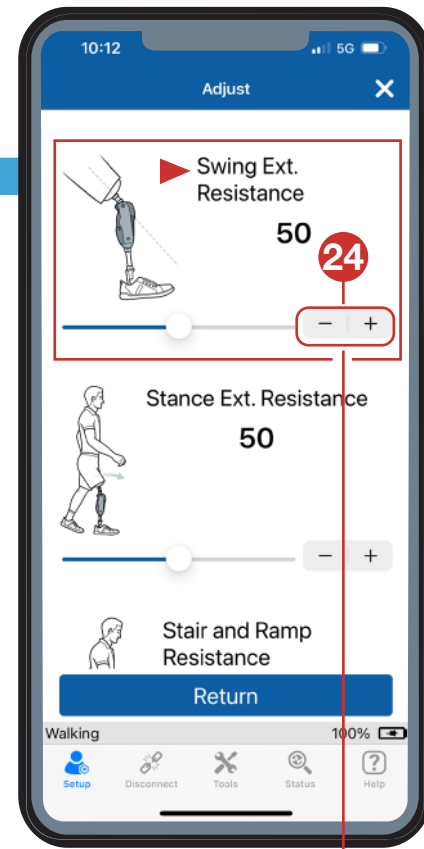Adjust



Controls **swing extension resistance**.

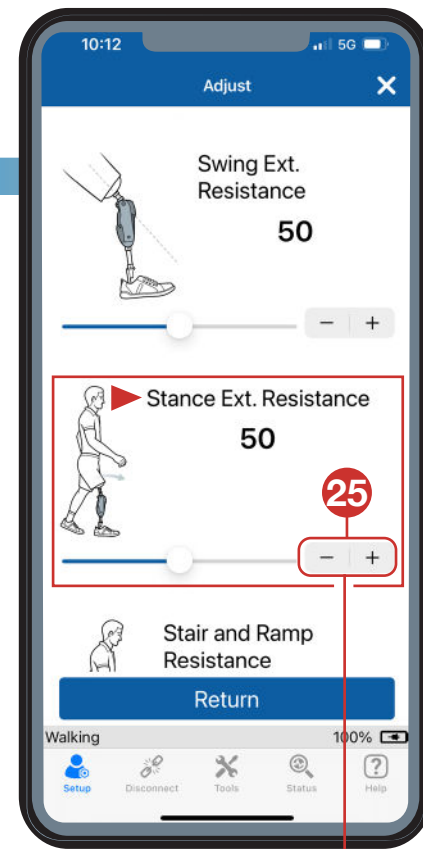- To decrease resistance
- + To increase resistance

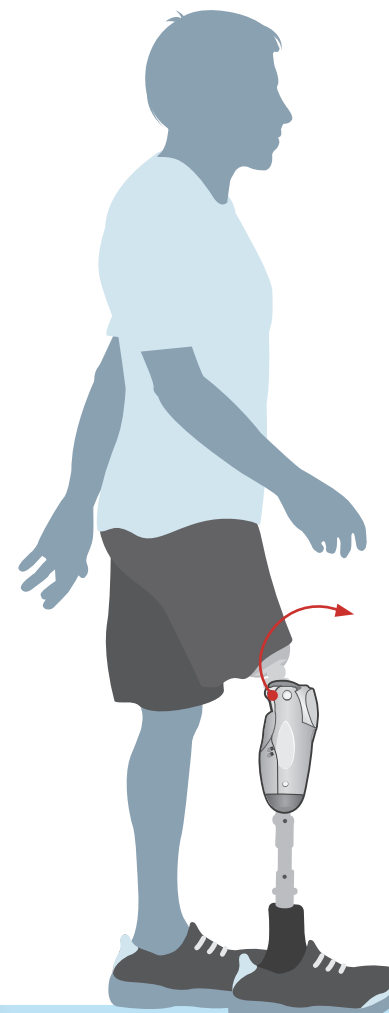
Controls **stance extension resistance**.


- To decrease resistance
- + To increase resistance

NOTE

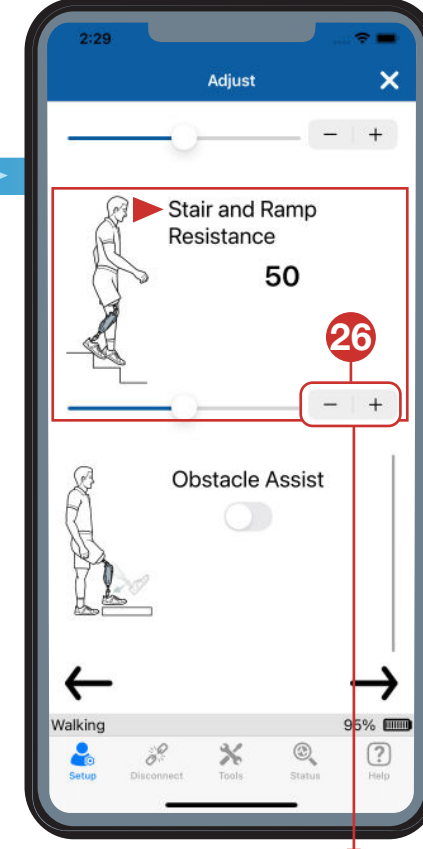
Only adjusted when walking with **Stance Flexion**.

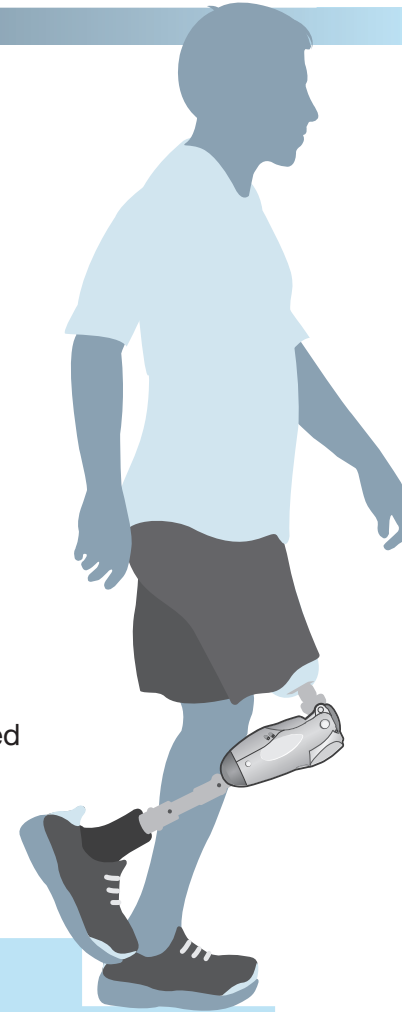
Provides independent resistance for **stair and ramp decent**.

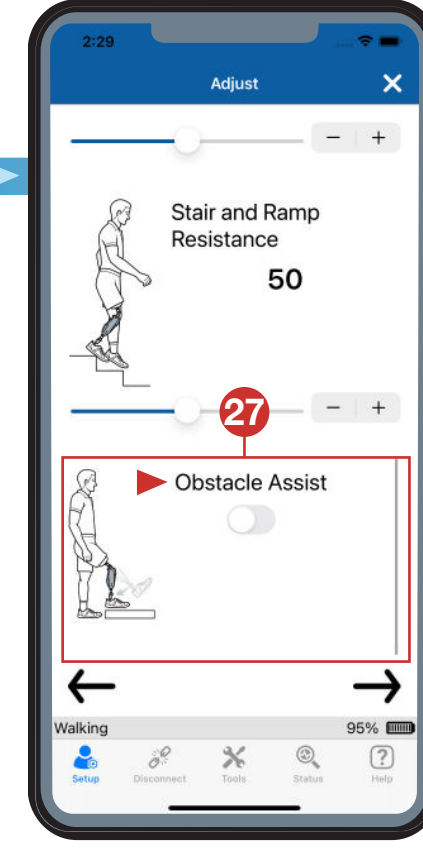- When knee flexion is delayed due to high resistance
- + When knee flexion is rapid due to low resistance

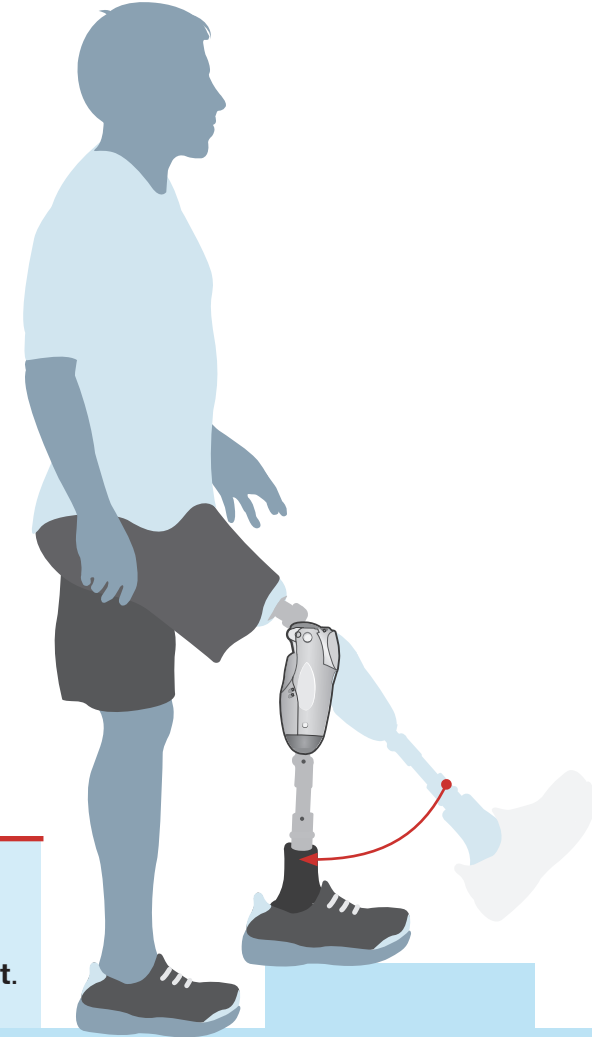Provides help encountering an **obstacle**.



NOTE

If **Sitting Mode** is turned off, it will also disable **Obstacle Assist**.




Demonstrate

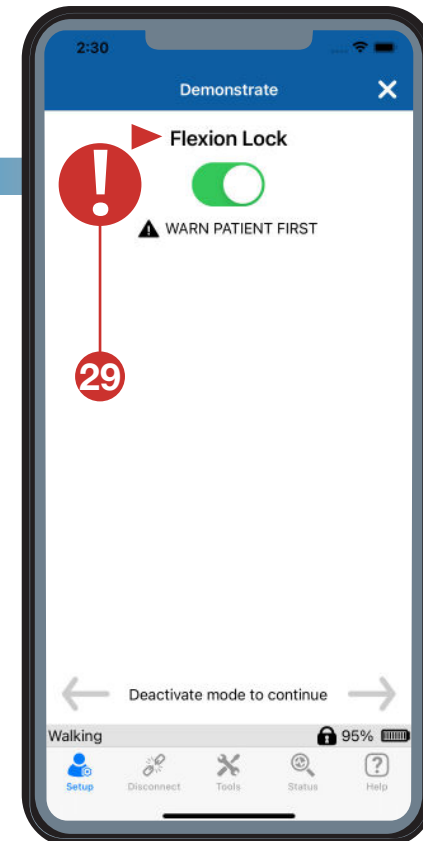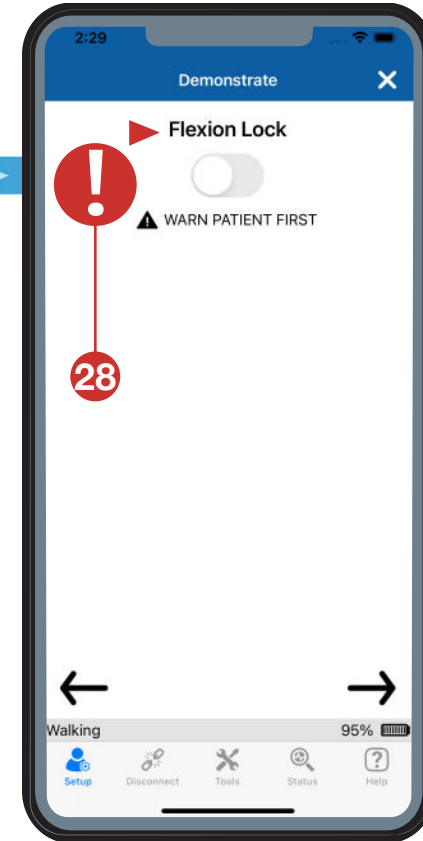When the **Flexion Lock** is engaged, it blocks flexion but allows ratcheting up to full extension.


Demonstrate and Adjust

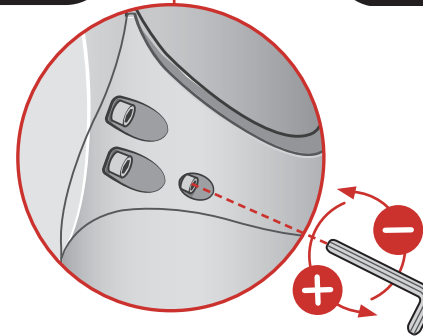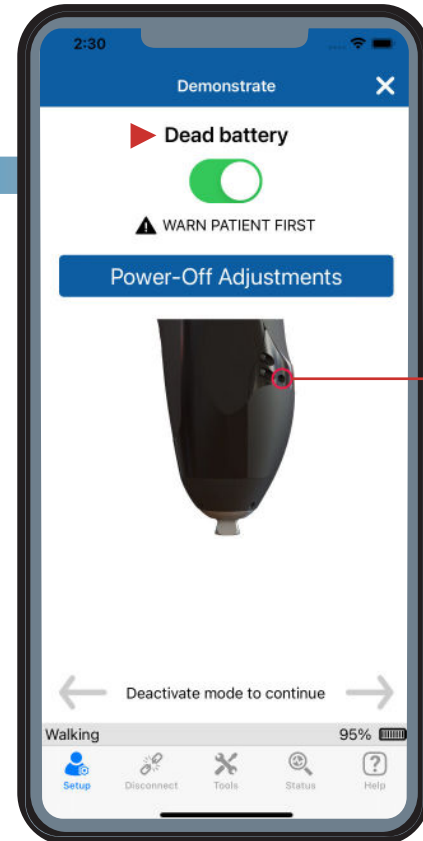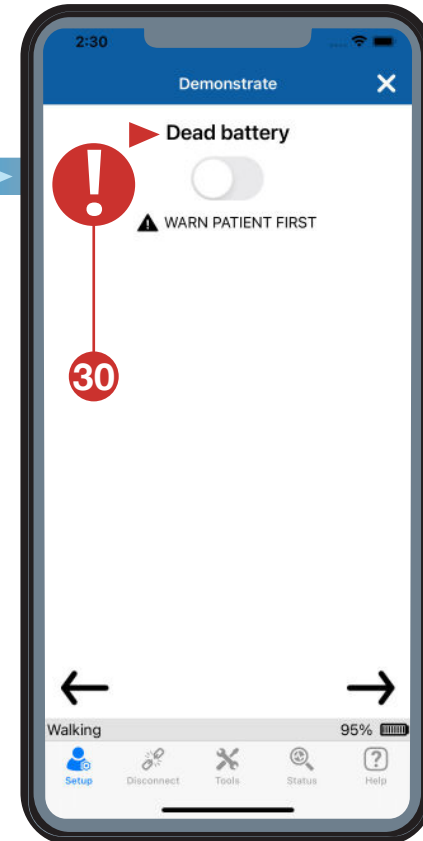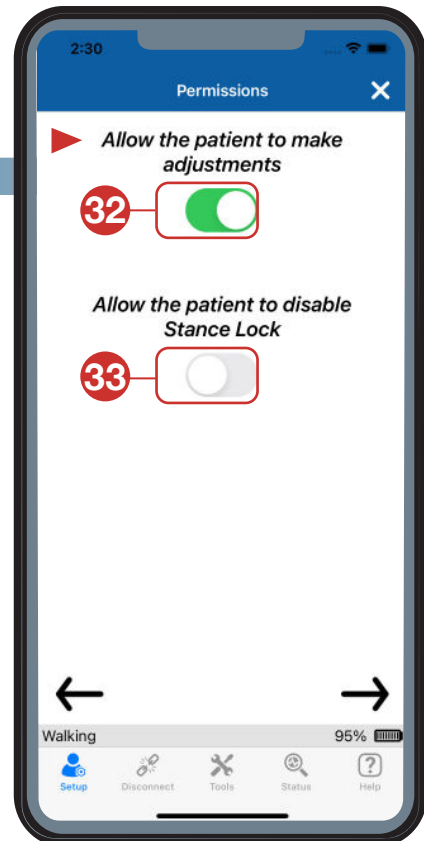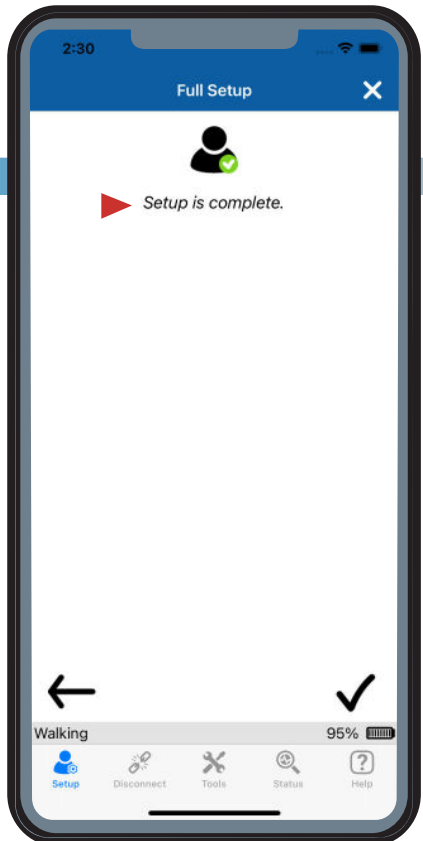
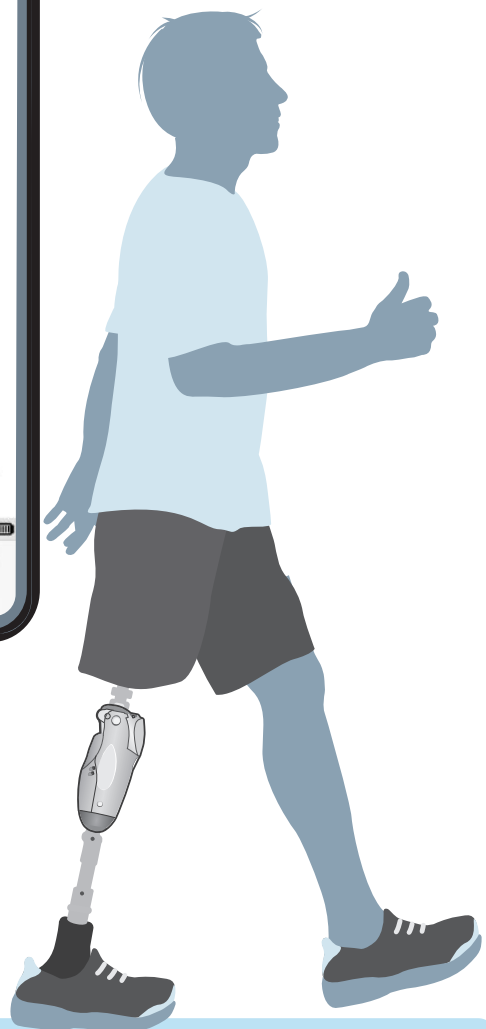
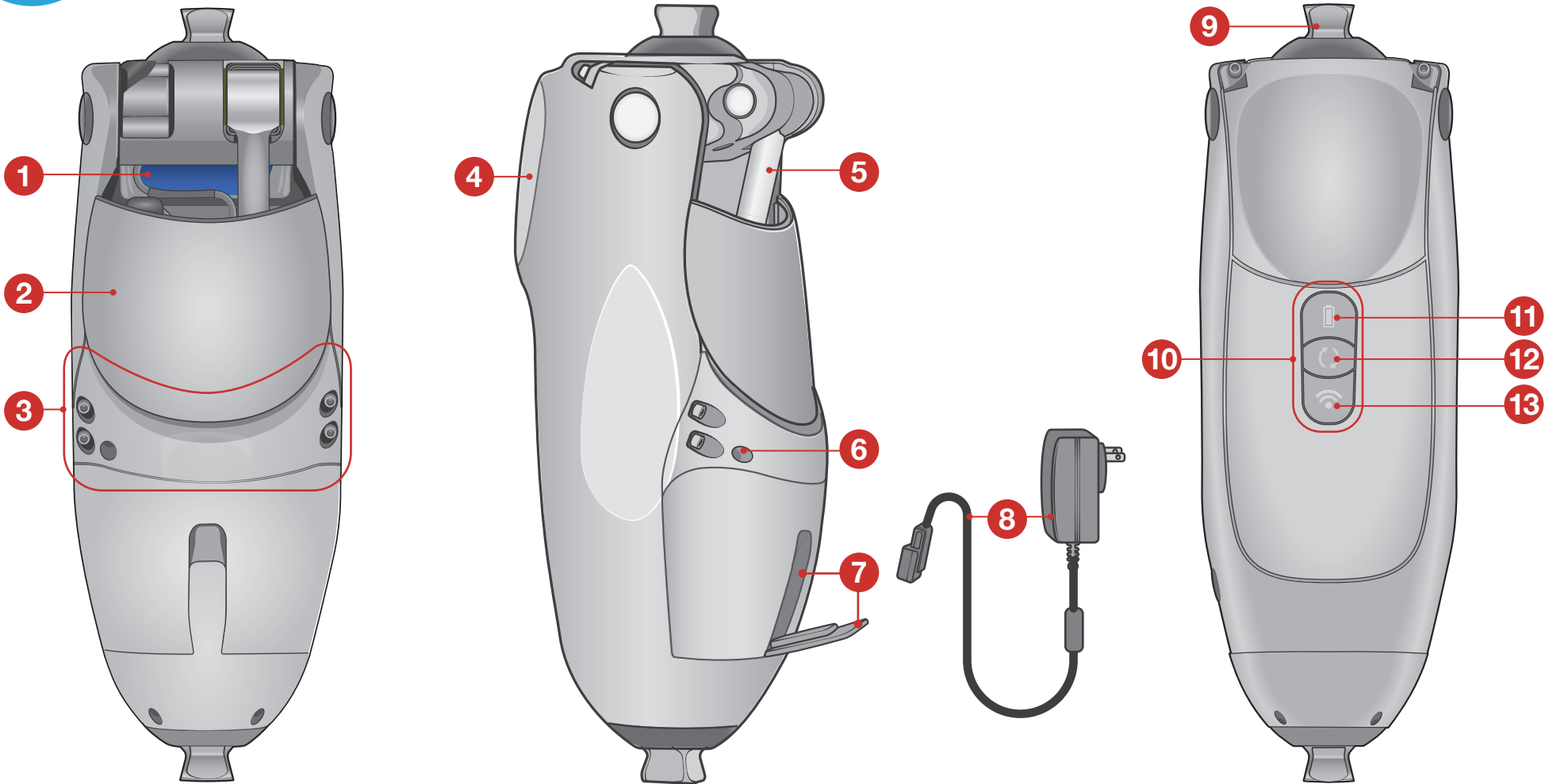
Perform resistance adjustments using a fully discharged battery.

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© 2026 by PROTEOR All rights reserved.
PROTEOR QUATTRO is a registered trademark of PROTEOR
© 2025 by PROTEOR All rights reserved.