PROTEOR

QUATTRO 2

MICROPROCESSOR KNEE

FR	Notice d'Utilisation
EN	Instruction for use
DE	Gebrauchsanweisung
IT	Istruzioni per l'uso
ES	Instrucciones de uso
PT	Instruções de utilização
NL	Gebruikershandleiding
DA	Brugervejledning
NO	Brukerveiledning
SV	Bruksanvisning
FI	Käyttöohjeet
PL	Instrukcja użytkowania
CS	Návod k použití
SK	Návod na používanie
HR	Upute za uporabu
RU	Инструкция
UK	Інструкція із застосування
JA	取扱説明書
ZH	使用说明
AR	مادخت سال تاميلت

Table des matières

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Table of contents

01 System Components	2
02 Terms / Symbols Used In This Guide.....	2
03 Warnings / Safety Precautions	2
04 Product Description	3
05 Indications	3
06 Intended Use.....	3
07 Contraindications	4
08 Physical Properties	4
09 Environmental	4
10 Basic Operation.....	4
10.1 Battery Information	4
10.2 Product details and interface.....	5
10.3 Charging Battery	5
10.4 Keypad Operation	6
10.5 Modes	7
10.6 Power-Off Behavior	9
11 Knee Cap Replacement.....	10
12 Flexion Angle Limiter Kit	10
13 Compatibility.....	11
14 Alignment and Setup	11
15 Cleaning	11
16 Transferring Settings/Resetting to Factory Defaults	11
17 Warranty and Service.....	12
18 Technical Support	12
19 Electromagnetic Compatibility Precautions	13
20 Guidance and Manufacturer's Declaration – Electromagnetic Emissions	13
21 Guidance and Manufacturer's Declaration – Electromagnetic Immunity.....	13
22 Regulatory compliance	14
Annex 1: Keypad Button Reference	15
Annex 2: LED Description	15
Annex 3: Change Mode Button Reference.....	15

01 | System Components

Part Description	Part Number	Sold Separately
PROTEOR QUATTRO 2 MPK - PYRAMID	QNX0010	N/A
PROTEOR QUATTRO 2 MPK - THREADED TOP	QNX0011	N/A
PROTEOR QUATTRO Battery Wall Charger & Adapters	QNX0601	No
Instructions for Use (IFU)	IFU-03-014	No
PROTEOR QUATTRO Booster Adapter	ACC0010	Yes
Kit, PROTEOR QUATTRO Knee Pad Replacement	ACC0003	Yes
Kit, PROTEOR QUATTRO Rear Cover Replacement	ACC0005	Yes
Kit, PROTEOR QUATTRO Rotor Plug Replacement	ACC0006	Yes
Flexion Angle Limiter Kit	ACC0007	No
Cover, Distal Pyramid	QNX2005	Yes

02 | Terms / Symbols Used In This Guide

	Type BF applied part		CE marking
	WEEE Waste Electrical and Electronic Equipment Should not be discarded as unsorted waste - must be sent to separate collection facilities for recovery and recycling		Battery/Power Status Button
	FCC mark		Change Mode Button
	Manufacturer		Wireless Button
	Serial number		Certified Prosthetist
	EU Authorized Representative		Refer to instruction manual / booklet
	Single patient, multiple use		Keypad Function
	QUATTRO MPC Knee System		

03 | Warnings / Safety Precautions

- Please read this instruction manual thoroughly before using.
- Failure to comply with safety precautions can result in QUATTRO malfunctioning and risk of injury to the user.
- Initial operation of QUATTRO should take place in the presence of a trained care professional.
- Using walking aids may lead to the misinterpretation of patient activity. If the patient typically uses walking aids, they must be utilized during the calibration process.

- Do not open or modify any component(s) and DO NOT make repairs on the QUATTRO unless otherwise directed in this IFU.
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and to the competent authority of the State in which the user is established.
- In case of damaged packaging, check there is no damage to the QUATTRO.
- If any issues or concerns with QUATTRO arise, contact your Prosthetist for assistance.
- Improper and unsafe use of QUATTRO may result in injury.
- Do not use the QUATTRO while the battery is charging via the wall charger. Be sure to disconnect from the charger prior to putting on the prosthesis.
- Do not submerge QUATTRO while charging with the QUATTRO Booster Adapter.
- If unusual movement or wear is detected in a structural part of the prosthesis at any time, immediately discontinue use of the device and consult your Prosthetist.
- QUATTRO contains a Lithium-ion battery that is not field replaceable and requires factory service for repair. Lithium-ion batteries contain hazardous metals and should never be disposed of in residential or commercial garbage. They should never be incinerated because they may explode. Reference local regulations for proper disposal of product.
- If power is lost from the QUATTRO, it will default to Power-Off Flexion resistance.
- When the battery is depleted, use extreme caution when descending ramps and stairs.
- QUATTRO contains small parts and cords that can be potential choking hazards.
- If the QUATTRO has been stored outside its Operating Temperature Range, allow the Knee to acclimate to the operating environment for a minimum of 30 minutes before use.
- Avoid running or other intense physical exercise above the specified operating temperature limit (45 °C [113 °F]). This can damage your QUATTRO.
- The proximal connection to the patient is the Applied Part of the device. All other external surfaces of the device, including the power supply and cord, are the Accessible Parts.
- Failure to comply with the Instruction for Use will void the warranty.

04 | Product Description

QUATTRO is a microprocessor controlled, single axis prosthetic knee joint system with internal sensors to control knee flexion and extension resistance.

05 | Indications

QUATTRO is indicated for use as a component in a prosthetic leg for individuals with unilateral or bilateral lower-limb amputations or limb deficiencies, including:

- Transfemoral amputation
- Knee disarticulation amputation
- Hip disarticulation amputation
- Congenital, lower-limb deficiencies

06 | Intended Use

QUATTRO is for SINGLE PATIENT use and intended for use by prosthesis wearers who would benefit from the safety, stability, and improved gait dynamics of a microprocessor-controlled knee. QUATTRO should only be fit by a trained Certified Prosthetist.

- Suitable for ambulatory level: K3/K4
- User Weight Limit:
 - 150 kg/330 lbs (K3)
 - 136 kg/300 lbs (K4)

ISO 10328 – P7 – R3 –   150 Kg

07 | Contraindications

QUATTRO is inappropriate for users with:

- Insufficient cognitive ability to charge the knee and care for the device
- Inability to walk with an unlocked knee

08 | Physical Properties

- Product Weight: 1655 g [3.65 lbs.]
- Build Height: 216 mm [8.5 in]
- Maximum Flexion Angle: 135°

09 | Environmental

- Operating Temperature Range: 0 to 45 °C [32 to 113 °F]
- Storage Temperature Range: -25 to 60 °C [-13 to 140 °F]
- Altitude Range: 0-5,000 m [0 -16,400 ft.]
- Humidity Range: 0% to 100%, including condensation
- Water Resistance: Rated to IP68 under IEC 60529
 - QUATTRO is safe for occasional submersion in fresh water up to 3 m [10 ft.] for up to 60 minutes. The user should dry the Knee with a towel once out of the water.
 - QUATTRO can be used for occasional showering with mild soaps and within operating temperatures of the knee.
 - QUATTRO should not be used for bathing.
 - QUATTRO should not be immersed in saltwater or chlorinated water as these may cause corrosion.
 - If QUATTRO is sprayed by or accidentally submerged in salt water or chlorinated water, immediately rinse with fresh water and towel dry.

10 | Basic Operation

10.1 Battery Information


Operating voltage: 7.4 VDC

Charger input voltage: 12 VDC


AC power adapter input voltage: 100-240 VAC, 50/60 Hz

Battery 2-3 days, plus an available Booster Adapter with additional days of battery based on the following user profile: sleeping, sitting, standing, walking, activity, driving.


10.2 Product details and interface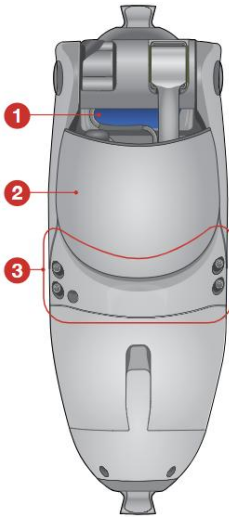

- 1. Flexion angle limiter
- 2. Flexible back cover
- 3. Rear brace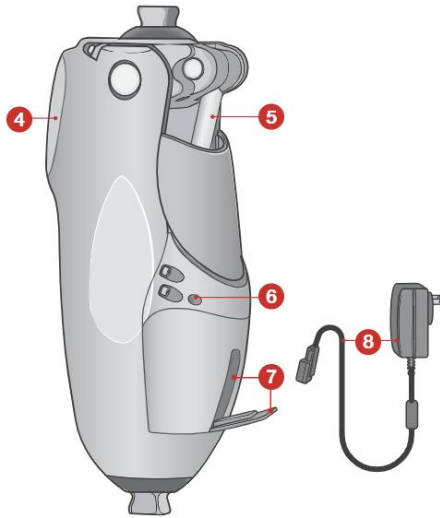

- 4. Replaceable knee pad
- 5. Cylinder shaft
- 6. Power Off Flexion Resistance adjustment screw
- 7. Charging port and dust cover
- 8. Battery charger and cable

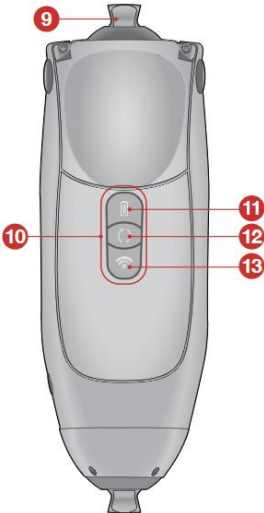- 9. Pyramid adapter
- 10. Keypad
- 11. Battery Status
- 12. Change Mode
- 13. Wireless

10.3 Charging Battery

Pull the protective charging port cover (7) open and insert charge connector (8) with the cable oriented up. The connector is magnetic and will snap into place. Once inserted, the battery LED button will illuminate briefly, indicating that the Knee is charging.

⚠ Use only the Battery Wall Charger & Adapters or the Booster Adapter that are designed for QUATTRO to charge the device.

⚠ Only use a 3mm hex wrench for making Power Off Flexion Resistance changes.

10.4 Keypad Operation

10.4.1 Battery Button (11)

Check battery status: Press and release button to show the current battery power status.

Status	Charge level	LED description
Fully charged	51-100%	Solid Green
Charged	21-50%	Flashing Green
Low	11-20%	Solid Red
Very low	1-10%	Flashing Red

Battery charging: When plugged into a charger, the current battery charge is indicated by a flashing **Green** light. The more the battery is charged, the longer each flash will stay lit.

Rebooting the knee: Hold the button for six seconds until the **Green** and **Blue** lights flash. Only use this feature as directed by your Prosthetist if you are having problems with your device.

10.4.2 Change Mode Button (12)

Function	Status	LED description
Check Activity: Press and release button to show current Activity	Walking	Slow flashing white
	Selected Alternate Activity	Quick flashing white
Enable/disable Flexion Lock: Double press the button to set or release the Flexion Lock.	Lock engaged	Solid white
	Lock engaged, with very low battery	Solid white Flashing Red
	Lock disengaged	Previous LED activity resumes
Change Activity: Hold the button for 3 seconds to switch back and forth between Walking and your currently selected Alternate Activity. When changed, lights will indicate new Activity.	Activity changed to Walking	Slow flashing white
	Activity changed to Alternate	Quick flashing white
	Activity did not change (Alternate not enabled in patient app or it is unsafe to change Activity right now)	Flashing white and Red

10.4.3 Wireless Button (13)

Pair Knee with Mobile App: To pair your QUATTRO to the Smart device the first time you connect, quickly press and release the button when prompted by app.

10.4.4 Put Knee into Storage Mode

Hold both the **Change Mode** and **Wireless** buttons at the same time for six seconds until all the lights flash and then turn off. When in Storage Mode the QUATTRO will be completely powered off and unresponsive. Pressing the Battery/Power button will not turn the device back on. To turn the Knee back on, you must plug it into the charger or Booster Adapter. The Knee is shipped in Storage Mode from the factory (must be plugged into power source to turn on).

NOTE: The device will not enter Storage Mode if connected to the charger.

10.5 Modes

Sitting Mode:

QUATTRO features two available behaviors when sitting, and the adjustment is found under the "Tools" tab along the toolbar in both the Clinician and Patient apps.

Sitting Mode On (default): When the Knee is unweighted and thigh is near the horizontal plane, the Knee flexes without resistance.

Sitting Mode Off: Knee will continue to have hydraulic resistance to flexion independent of thigh flexion angle.

Note: If Sitting Mode is turned off, it will also disable Cycling and Obstacle Assist.

Obstacle Assist:

QUATTRO's Obstacle Assist feature is helpful for stepping over tall objects or when entering a high vehicle.

Obstacle Assist On (default): The Knee will flex with no resistance when the thigh reaches 35 degrees of flexion or greater and is extended and unweighted.

Obstacle Assist Off: The Knee will flex slowly with hydraulic resistance when unweighted independent of thigh flexion angle.

The Clinician can turn this feature off/on for different Activities for the patient under "Setup" and "Adjust" along the toolbar in the GaitLab app after full programming is completed.

The Clinician must give a patient permission to turn this feature on/off. It can be found under "Setup" and then "Permissions" along the toolbar of the Clinician app.

If patient has permission for this feature, it can be found under "Tools" and then "Settings" along the toolbar of the Patient app.

Note: If Sitting Mode is turned off, it will also disable Obstacle Assist.

Cycling:

In Cycling, the Knee will enter a user-defined low resistance setting when the user's thigh is near the horizontal plane which occurs when getting on the bike or during normal pedaling. The user can define three flexion/extension resistances in the app using "+" or "-". The Knee will remain in low resistance until the user puts weight on the Knee in full extension, triggering the QUATTRO to return to Walking function. When the Knee is in Cycling, it will disable Obstacle Assist, Sitting Mode, and Stance Lock.

*It is not recommended to keep Knee in Cycling for extended periods of time after patient has finished the Activity. Use app or Keypad to switch Activity back to Walking to avoid triggering Cycling unintentionally.

Note: If Sitting Mode is turned off, it will also disable Cycling.

Elliptical:

Elliptical was validated for unilateral QUATTRO users for use on elliptical machines. There is a single Resistance setting for both flexion and extension, and users can pedal forward and backwards on the machine. Stance Lock and Sitting Mode are disabled when in Elliptical. It is not recommended for stairs or stepping machines, to be left in Elliptical at all times, and caution must be used when walking backwards and down slopes in Elliptical.

Golfing:

Golfing allows you to limit the amount your Knee flexes while playing golf.

After selecting Golfing on the Activities screen via the Patient app, navigate to the Adjustment Screen. On this screen you will have 3 choices for different degrees of flexion that will be allowed. Each of these can be customized by pressing the "+" or "-". The patient can program up to 3 different custom resistance settings for the amount of flexion needed for either a particular golf swing or other activities requiring custom resistance settings.

*It is not recommended to keep Knee in Golfing for extended periods of time after patient has finished the Activity. Use app or Keypad to switch Activity back to Walking to avoid triggering Golfing unintentionally.

⚠ WHEN GOLFING IS ENABLED, THE KNEE WILL ONLY FLEX TO THE DEGREE THAT HAS BEEN CHOSEN IN THE APP. DESCENDING STAIRS AND SLOPES SHOULD BE AVOIDED WHEN GOLFING IS ENABLED.

Rehabilitation:

Rehabilitation was designed for unilateral and bilateral QUATTRO users for use during rehabilitation. It provides a Sit-to-Stand Support Lock with a Sitting Resistance setting separate from the Walking Sitting Resistance, and a Sit-to-Stand Support Timeout in seconds. When in Rehabilitation, when the user rises from the chair but stops ascending, the knee locks to flexion at that angle. If the set Timeout time is reached and the user has not extended the knee more, it unlocks flexion and lowers the user into the chair using the set Rehabilitation Sitting Resistance. If the knee is extended again before the Timeout time is reached, the knee locks flexion at the current angle and the Timeout timer restarts at zero. Exit Rehabilitation by fully extending the knee. It will be in Walking resistances until the user sits again, then the Sit-to-Stand Support Lock will automatically reactivate. The knee can be left in Rehabilitation all the time as long as the user is informed that Sit-to-Stand Support Lock will activate each time they take a seat.

Sprinting:

Sprinting was validated for unilateral QUATTRO users for use during sprinting activities. Obstacle Assist and Sitting Mode are disabled when in Sprinting. All resistance settings can be adjusted in Sprinting to accommodate each user's needs. It is not recommended to be left in Sprinting all the time.

Stair Ascent:

Stair Ascent was validated for unilateral QUATTRO users for use during ascending of steps using step-over-step gait. This function is not for use by users with hip disarticulation or insufficient hip joint or pelvic voluntary muscle control.

Stair Ascent is Off by default, requiring the Clinician to enable it in the Clinician app in the "Tools" tab on the Settings screen. When enabled, Stair Ascent has a dedicated Swing Extension Resistance that can be adjusted.

The Clinician must give the patient permission to turn this feature on/off and adjust the Swing Extension Resistance. It can be found under "Setup" and then "Permissions" along the toolbar of the Clinician app.

If patient has permission for this feature, it can be found under "Tools" and then "Settings" along the toolbar of the Patient app.

When Stair Ascent is enabled, the patient must perform a rapid hip extension then flexion movement with no load on the prosthesis to initiate Stair Ascent. The knee flexes to provide clearance over the step and ensures the foot is flat when landing on the next step. When the knee is loaded, it is locked in flexion but free to extend.

Stance Lock:

QUATTRO includes a Stance Lock feature for stability that will automatically lock when patient is on a flexed Knee.

Stance Lock On (default): When the patient stops the thigh at a flexion angle and weights the Knee, it will automatically lock the Knee from further flexion. The lock will automatically disengage when the thigh moves in an extension moment, or the Knee is unweighted.

Stance Lock Off: When the patient stops the thigh at a flexion angle and weights the Knee, the Knee will not lock and it will allow the Knee to continue to flex.

The Clinician must give a patient permission to turn this feature on/off and it can be found under "Setup" and then "Permissions" along the toolbar of the GaitLab app.

If patient has permission to turn this feature on/off it can be found under "Tools" and then "Settings" along the toolbar of the patient app.

Note: Stance Lock is available from 5-70° of flexion. If the Knee is in Cycling, Elliptical or Golfing it will disable Stance Lock.

Flexion Lock:

To engage Flexion Lock while in any Activity, double click the Change Mode button. The button will light up with a solid white light to indicate that Flexion Lock is engaged. When the lock is engaged, the Knee can straighten but it cannot bend any further than the current angle. If the Knee is straightened to a more upright position, the lock re-engages at the new angle. The Knee will remain locked at this position until the knee is straightened, or the Flexion Lock is disengaged. The solid white light remains on as long as the Flexion Lock is engaged.

Note: When Flexion Lock is engaged and the Knee is unweighted, it will be locked in both flexion and extension.

With Flexion Lock engaged, double click the Change Mode button to disengage. The solid white light now turns off, indicating that the Flexion Lock has been disengaged. The Knee returns to the currently selected Activity, e.g., Walking.

⚠️ SOCKETS AND COMPONENTS MUST NOT CONTACT THE FRAME OR CYLINDER. THIS WILL CAUSE DAMAGE TO THE UNIT. CONTACT BETWEEN THE SOCKET AND QUATTRO'S REAR BRACE (NO. 3 ON PRODUCT DETAILS) IN FULL FLEXION IS RECOMMENDED AND WILL NOT CAUSE DAMAGE.

10.6 Power-Off Behavior

When the battery is depleted or the Knee has been put into Storage Mode, a Power-off Flexion resistance is used. The Power-off Flexion resistance can be adjusted to accommodate different user preferences and weights.

⚠️ PLEASE HAVE YOUR PROSTHETIST ADJUST THIS SETTING FOR YOU.

Making an Adjustment:

With the QUATTRO Knee fully extended, insert a 3mm hex wrench into the Power-Off Flexion resistance screw (see no. 6 on product details): CW  adjustments provide more flexion resistance and CCW  adjustments provide less flexion resistance.

⚠ DO NOT FLEX THE KNEE WITH THE HEX WRENCH INSERTED, THIS MAY CAUSE PERMANENT DAMAGE TO YOUR DEVICE.

Note: The Knee ships with the flexion resistance at the middle of the range, plus 2 turns, to allow for increased or decreased resistance based on patient preference.

11 | Knee Cap Replacement

Removing the Knee Cap:

Use a 3/32" Hex Key to remove the two socket head cap screws from the Knee.

Installing the Knee Cap:

Install Knee Cap. Tighten the two socket head cap screws using the 3/32" Hex Key.

Note: Tighten until flush plus 1/4 turn, do not over torque screws.

12 | Flexion Angle Limiter Kit

1. Ensure the Knee is in full extension.
2. Use a 3/32" Hex Key or Driver to remove the two socket head cap screws from the rear of the Knee.
3. Maneuver Flexion Angle Limiter around the Cylinder Shaft to remove.

Note: This may require orienting the back of the Knee downward.

4. Ensure the Knee is in full extension.
5. Maneuver the replacement Flexion Angle Limiter into position to install.
6. Tighten the two socket head cap screws using a 3/32" Hex Key or Driver.
7. The inscribed text on the Flexion Angle Limiter should be visible.

Note: Tighten until flush plus 1/2 turn (equivalent to 80oz-in torque).

13 | Compatibility

QUATTRO is available with a male pyramid connection or threaded top connection designed to be compatible with standard connectors (see our catalog).

Use only the Battery Wall Charger & Adapters provided with QUATTRO or the optional Booster Adapter for battery charging.

14 | Alignment and Setup

Download the Clinician GaitLab smartphone app to properly align and setup the device. GaitLab is required to calibrate and optimize the microprocessor settings for each patient. Patients can change Modes and adjust settings with the Freedom Innovations App. Verify the smartphone requirements before app installation. Download Apps from the App Store or Google Play Store.

The GaitLab and Freedom Innovations smartphone apps are compatible with iOS 17 or later and Android OS 14 or later.

App		
GaitLab 		
Freedom Innovations 		

15 | Cleaning

Use clean tap water to rinse off your QUATTRO Knee. A soft-bristled brush (such as a toothbrush) is acceptable to dislodge stuck dirt on the outside of the Knee only. Using any other chemicals or soaps on the QUATTRO may void the warranty. Dry the Knee thoroughly with a towel after rinsing. Do not use a hairdryer, which may damage the electronics and void the warranty.

16 | Transferring Settings/Resetting to Factory Defaults

Transferring Settings:

1. Connect to the Knee you'd like to transfer the program from.
2. Once connected, go to the "Tools" tab along the toolbar of the app, then tap "Manage Settings".

3. Tap "Copy settings" to save the settings to your device.
4. You can now disconnect from the programmed Knee.
5. Connect to the Knee you'd like to transfer the program to.
6. Go to the "Tools" tab along the toolbar of the app, tap "Manage Settings", and tap "Paste settings".
7. You will get a confirmation message "Are you sure you want to replace all settings?" Tap "OK" to transfer all settings, or "Cancel" to cancel the paste and keep the settings unchanged.
8. Calibration does not transfer over and will need to be recalibrated.

Reset to Factory Defaults:

This will reset all the settings in the Knee back to factory settings. This action cannot be undone.

1. Connect to the Knee you'd like to reset the settings for.
2. Once connected, go to the "Tools" tab along the toolbar of the app, then tap "Reset to Factory Defaults."
3. You will get a confirmation message "Are you sure you want to reset all settings on your QUATTRO to factory defaults..."
4. Tap "OK" to reset the Knee, or "Cancel" to exit.

 Recommendation: Have the patient sitting while transferring the program or resetting the Knee for safety.

17 | Warranty and Service

QUATTRO includes a 36-month warranty covering all manufacturer defects effective only if the product is used according to manufacturer's recommendations. QUATTRO does not need to be serviced during this time. Service and maintenance are required at 36-months if the optional 2-year extended warranty is purchased; and, at 36-months and 54-months if the optional 3-year extended warranty is purchased. The battery charger and accessories are provided with a 12-month warranty. The lifetime of QUATTRO depends upon the patient's level of activity. See the company website for details.

For Knee service or repair, please contact PROTEOR. A loaner Knee will be provided upon request. For loaner and shipping information please see the Technical Support section.

18 | Technical Support

- For User Support, please contact your Certified Prosthetist.
- For Prosthetist Technical Support, supplementary training materials or to schedule service, please contact PROTEOR.
- A loaner Knee can be provided upon request. Knees requiring service should be shipped in the provided case to the following address (outside of Europe):
 PROTEOR USA, LLC
 491 East 350 North
 Gunnison, Utah 84634, USA
- In Europe, please ship to the following address:
 PROTEOR SAS
 Rue du Cheffin
 21250 Seurre, France

19 | Electromagnetic Compatibility Precautions

QUATTRO needs special precautions regarding electromagnetic compatibility (EMC). Specifically, it needs to be installed and put into service according to the EMC information provided as follows:

- QUATTRO may be susceptible to electromagnetic interference from portable and mobile RF communications devices such as mobile (cellular) telephones.
- QUATTRO may be interfered with by other equipment, even if that other equipment complies with CISPR EMISSION requirements.
- QUATTRO complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:
 - (1) QUATTRO may not cause harmful interference, and
 - (2) QUATTRO must accept any interference received, including interference that may cause undesired operation.

20 | Guidance and Manufacturer's Declaration – Electromagnetic Emissions

QUATTRO is intended for use in the electromagnetic environment specified below. The customer or user of QUATTRO should ensure that it is used in such an environment.

Emissions Test	Compliance	Electromagnetic Environment- Guidance
RF Emissions EN 55011/ CISPR	Group 1	QUATTRO uses RF energy only for its CISPR internal function. Therefore, its RF emissions are low and are not likely to cause any interference in nearby electronic equipment.
RF Emissions EN 55011/ CISPR	Class B	QUATTRO is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonics IEC 61000-3-2	Complies	
Harmonics IEC 61000-3-2	Complies	

21 | Guidance and Manufacturer's Declaration – Electromagnetic Immunity

QUATTRO is intended for use in both a professional and home healthcare environment. The customer or user of QUATTRO should ensure the electromagnetic compliance of the environment.

Immunity Test	IEC 60601 Test Level	Compliance Level
Electrostatic Discharge IEC 61000-4-2	±8kV Contact ±15kV Air	±8kV Contact ±15kV Air
Radiated radio frequency electromagnetic fields IEC 61000-4-3	10 V/m 80 MHz-2.7 GHz 80% AM at 1 kHz	10 V/m 80 MHz-2.7 GHz 80% AM at 1 kHz
Electrical fast burst transients IEC 61000-4-4	±2kV Mains ±1kV I/Os	±2kV Mains N/A
Surge IEC 61000-4-5	±1kV Differential ±2kV Common	±1kV Differential ±2kV Common

Immunity Test	IEC 60601 Test Level	Compliance Level
Conducted disturbances induced by RF fields IEC 61000-4-6	3 V 0.15 MHz-80 MHz 6 V ¹⁾ in ISM between 0.15 MHz and 80 MHz ²⁾	3 V 0.15 MHz-80 MHz 6 V ¹⁾ in ISM between 0.15 MHz and 80 MHz ²⁾
Magnetic field at network frequency IEC 61000-4-8	30 A/m	30 A/m
Voltage dips and interruptions IEC 61000-4-11	>95% dip for 0.5 cycle >95% dip for 1 cycle 30% dip for 25/30 cycles >95% dip for 250/350 cycles	>95% dip for 0.5 cycle >95% dip for 1 cycle 30% dip for 25/30 cycles >95% dip for 250/350 cycles

¹⁾ r.m.s. before modulation is applied.

²⁾ The ISM (industrial, scientific and medical) bands between 0,15 MHz and 80 MHz are 6,765 MHz to 6,795 MHz; 13,553 MHz to 13,567 MHz; 26,957 MHz to 27,283 MHz; and 40,66 MHz to 40,70 MHz. The amateur radio bands between 0, 15 MHz and 80 MHz are 1,8 MHz to 2,0 MHz, 3,5 MHz to 4,0 MHz, 5,3 MHz to 5,4 MHz, 7 MHz to 7,3 MHz, 10,1 MHz to 10,15 MHz, 14 MHz to 14,2 MHz, 18,07 MHz to 18,17 MHz, 21,0 MHz to 21,4 MHz, 24,89 MHz to 24,99 MHz, 28,0 MHz to 29,7 MHz and 50,0 MHz to 54,0 MHz.

Test Frequency (MHz)	Band (MHz)	Modulation	Required Level	Compliance Level
385	380-390	18 Hz Pulse ¹⁾	27	27
450	430-470	18 Hz Pulse ¹⁾	18	28
710/745/780	704-787	217 Hz Pulse ¹⁾	9	9
810/870/930	800-960	18 Hz Pulse ¹⁾	28	28
1720/1845/1970	1700-1900	217 Hz Pulse ¹⁾	28	28
2450	2400-2570	217 Hz Pulse ¹⁾	28	28
5240/5500/5785	5100-5800	217 Hz Pulse ¹⁾	9	9

¹⁾ Carrier shall be modulated using a 50% duty cycle square wave signal.

22 | Regulatory compliance

QUATTRO is a CE-marked medical device and is certified as conforming with Regulation (EU) 2017/745, EN 55011 Class B, IEC 60601-1, ETSI EN 300-328 under Directive 2014/53/EU and ISO 10328. The device complies with Part 15 of the FCC Rules.

Annex 1: Keypad Button Reference

Feature	Action	Hold Time
Check Battery Status	Press Battery Button	—
Flexion Lock Enable	Double Press Change Mode Button	—
Flexion Lock Disable	Double Press Change Mode Button	—
Check Activity	Press Change Mode Button	—
Change Activity	Hold Change Mode Button	3 sec
Pair	Press Wireless Button	—
Storage Mode	Hold Change Mode + Wireless Buttons	6 sec
Reset	Hold Battery Button	6 sec

Annex 2: LED Description

Feature	LED Description
Battery is Charged	Solid Green
Battery is Charging	Flashing Green
Battery is Low	Solid Red
Battery is Very Low	Flashing Red
Knee is Manually Locked	Solid White
Activity Checked or Change	Flashing White: slow (Walking); fast (Alternate)
Activity Did Not Change	Alternate Red / White
Entering Storage Mode	Flashing Red / Blue / White
System Error	Alternate Red / Blue
Bootng	Flashing Green / Blue

Annex 3: Change Mode Button Reference

User Press	Current State	Product Behavior
Single Press to Check Status	Unlocked	White LED blinks to indicate current Activity. Slow blink indicates Walking. Fast blink indicates Alternate.
	Locked	LEDs blink Red / White to indicate error. No Activity is active to check because the Knee is Locked.
Double Press to Enable / Disable Lock	Unlocked	Flexion Lock is engaged. White LED stays solid.
	Locked	Flexion Lock is disengaged. White LED stays off. Previously selected Activity is resumed.
Long Press to Toggle between Walking and	Current activity is Walking but an Alternate has not been configured.	LED blinks Red / White to indicate nothing has happened.

User Press	Current State	Product Behavior
Alternate Activity	Current activity is Walking, and an Alternate has been configured.	Alternate activity is selected. White LED blinks quickly to indicate Alternate is now selected.
	Current activity is Alternate.	Walking activity is selected. White LED blinks slow to indicate "Walking" is now selected.

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©2026 PROTEOR. All rights reserved.
PROTEOR QUATTRO is a registered trademark of PROTEOR.